ENGLISH

Instructions for use:

7. Attach the nonsterile Foam Insert (AR-1615) to the upper arm cradle.

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© 2020, Arthrex Inc. All rights reserved. DFU-0340-SUB, Revision 0 06/2020

INSTRUCTIONS
FOR USE (AR-1611S)

ARTHREX
WRIST
TRACTION
TOWER

Assembly
Montage
Ensamblaje
Assemblage
Assemblaggio

ENGLISH

The Wrist Traction Tower facilitates arthroscopic surgery of the wrist by maintaining the arm in a position comfortable for the patient, as well as accessible to the surgeon. Traction forces control wrist distraction, through the sterile Finger Traps, by using weights applied to the main boom cable. In addition to providing a counter-traction force, the upper arm cradle post will help maintain a stable position of the wrist for operative procedures such as reduction of distal radius fractures and fixation with pins or external fixation. The radiolucent hand stabilizer post is fully adjustable to firmly hold the wrist in any position from full pronation to full supination.

The radiolucent hand stabilizer post assembly can be removed from the main cross bar for autoclaving. The position of the upper arm cradle post and the radiolucent hand stabilizer post can be adjusted while the patient is in the supine position. *Note: The upper arm cradle and foam insert are nonsterile and must be draped during surgery.*

Sterile Finger Traps are secured in the desired fashion and hooked onto the triangular finger distraction attachment. Traction weights are then applied to a cable which is attached to the distal end of the finger distraction attachment.

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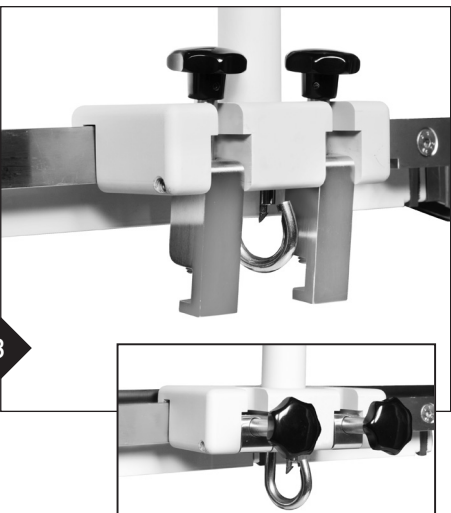
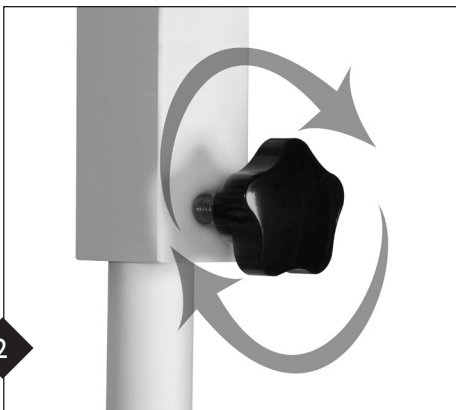
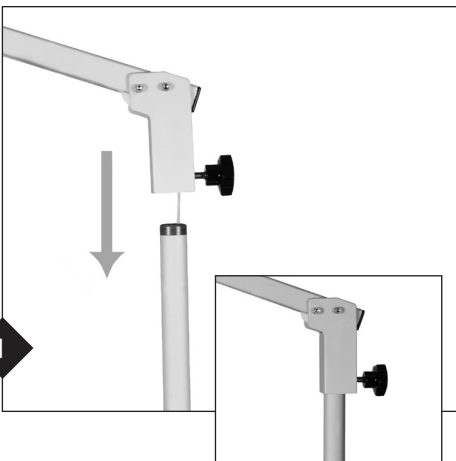
ENGLISH

Instructions for use:

1. Slide the boom cable down into the vertical upright beam. Seat the beam up inside the boom hinge.

2. Once assembled, swing the boom to the desired radial position and turn the knob clockwise to lock it into position.

3. Place the upper lip of the Clark Rail Adapter over the Clark Rail. Flip the base teeth up and tighten the two knobs for firm fixation to the Clark Rail.

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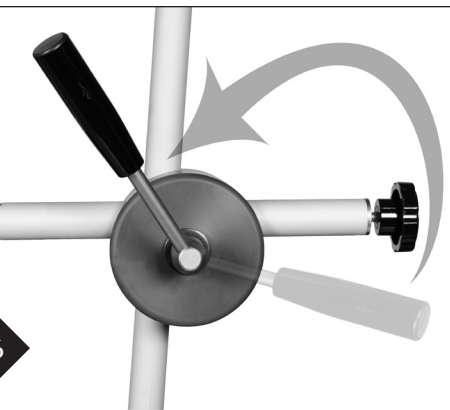
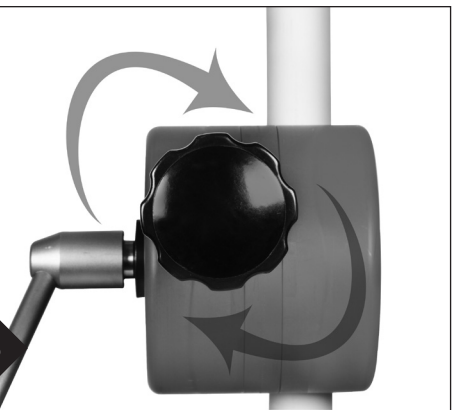
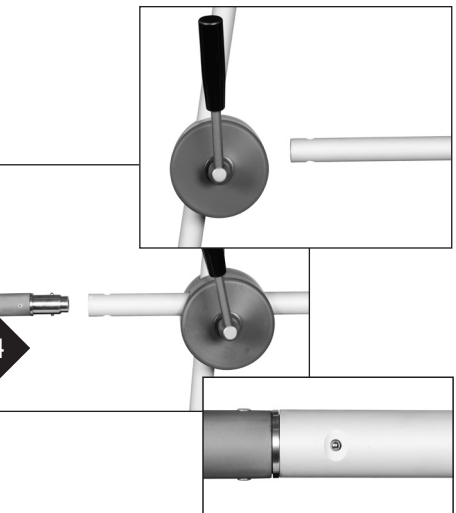
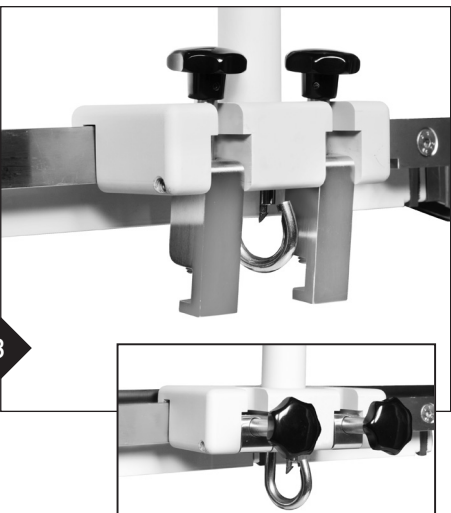
4. Insert the autoclavable, radiolucent hand stabilizer post into the adjustable horizontal arm. While inserting, be sure to line up the two snap buttons with the correlating holes in the horizontal adjustment arm. The post will snap into the horizontal adjustment arm. See Arthrex DFU-0023-XX for autoclaving instructions.

5. Tighten the knob in the clockwise direction to lock the post into the adjustable horizontal arm.

6. Loosen the adjusting handles counterclockwise and adjust the arms to the desired position. Both arms can be adjusted in or out, up or down, and side to side. *Note: The position of the levers is adjustable by placing your thumb on the screw post, pulling the lever out from the post, and turning the lever to the left or to the right.*

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