


Trans-SAP Approach for Cranial Extrusion


Quick Reference Guide

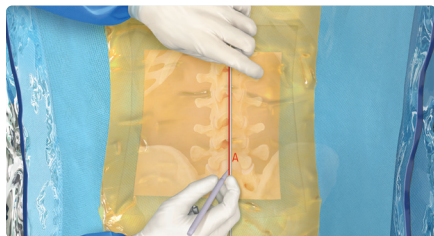

A

Midline


- With a blunt dissector, mark the midline using the spinous processes as reference (Line A).

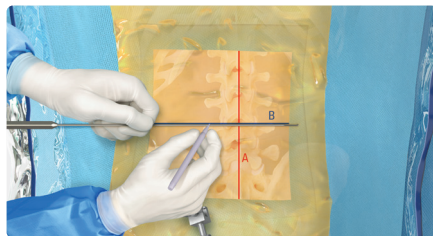

B

Disc Space


- Mark the disc space at the level of herniation (Line B).

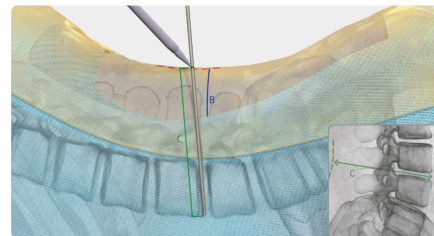

C


Measuring Lateral Line

- On a lateral x-ray, measure the distance from the ventral disc space to the dorsal surface of the skin (Line C).

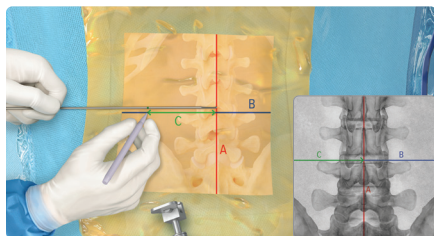

Marking Lateral Line

- From the midline (Line A), mark a horizontal line that is the length of Line C and aligned with the disc space line (Line B).

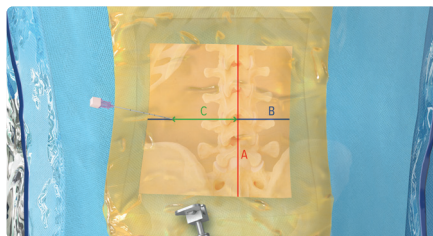


Needle Placement

- Insert the needle at the lateral end of Line C, advancing it along the disc space line (Line B) to dock on the ventral medial aspect of the superior articular process (SAP).

Far Lateral Discectomy L5-S1


Quick Reference Guide

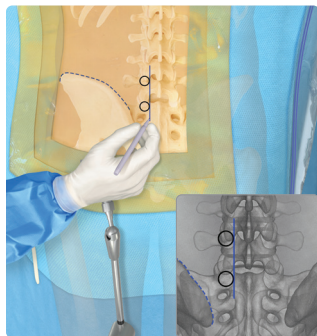

1

Crest and Pedicular Line


- ▶ Obtain an AP view of the S1 endplate
- ▶ Identify and mark the iliac crest and the medial border of the L5 and S1 pedicles

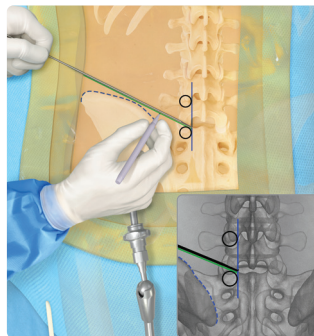

2

Oblique Line


- ▶ With a blunt dissector, mark an oblique line extending cranially from the pedicular line of the S1 pedicle, cranial to the iliac crest shadow

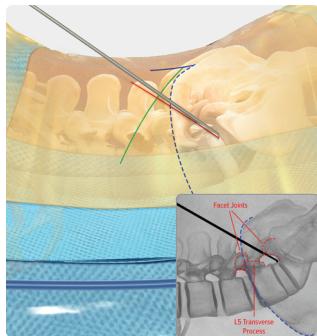

3

Lateral Line

- ▶ Mark a line starting at the junction of the S1 pedicle and endplate
- ▶ Ensure the trajectory passes through the pars interarticularis, cranial to the L5-S1 facet joint and caudal to the L5 transverse process

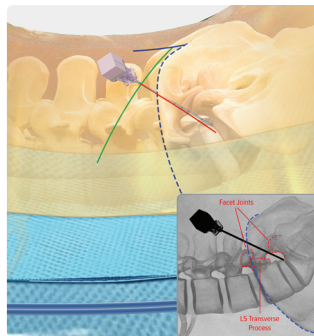


4

Needle Placement

- ▶ Use the intersection of the oblique and lateral lines as the insertion site for the needle, targeting the 12 o'clock position of the S1 pedicle using lateral and AP to confirm

Lateral Recess Decompression


Quick Reference Guide

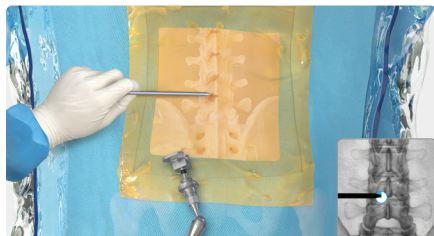

1

Target


- Tilt the C-arm to maximize the interlaminar window at the level of the decompression.
- In the interlaminar window, target between the 9 and 12 o'clock positions for a left-sided decompression or between the 12 and 3 o'clock positions for a right-sided decompression.

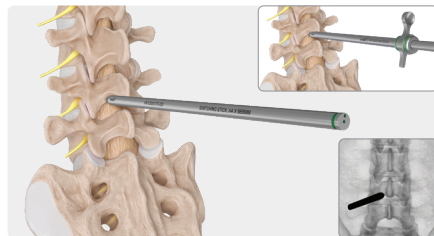

2

Access


- Use a #11 or #15 blade to create an incision <1 cm in length through the skin and fascia. Dilate through the musculature and fascia.
- Dock on the inferior medial edge of the cranial lamina. Use fluoroscopy to confirm positioning.

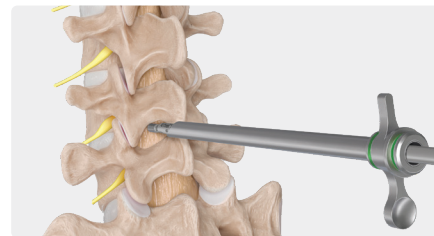

3

Expose


- Dissect to expose the lamina and ligamentum flavum, making sure to expose the junction between them.
- Rest the bur on the ligamentum flavum caudal to the cranial lamina.

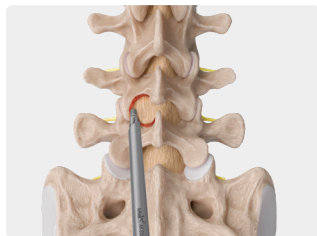

4

Undercut

- Undercut the ventral lamina at the attachment of the ligamentum flavum until the ligamentum flavum is detached.
- Follow the lateral aspect of cranial lamina to the medial facet and the caudal lamina. Rest the bur on the ligamentum flavum and bur the caudal lamina until the ligamentum flavum is detached.
- Extend this drilling to the medial facet until reaching the medial aspect of the superior articulating process.

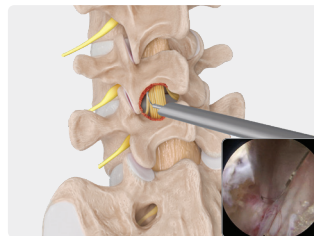


5

Resect and Decompress

- Resect the ligamentum flavum to decompress the traversing nerve. Decompress the lateral recess and mobilize the traversing nerve.

Sacroiliac Joint Denervation


Quick Reference Guide

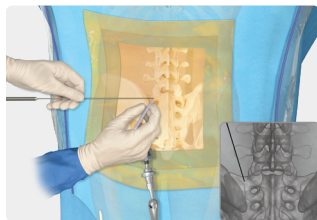

1

Targeting the L5 Medial Branch and S1 Lateral Branch


- Use an orthogonal AP view of the sacrum
- Identify the junction of S1 (SAP) and sacral ala and insert the needle and guidewire
- Make an incision for the cannula

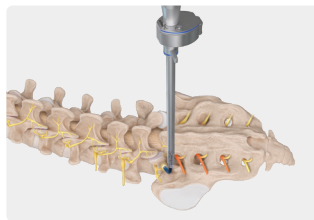

2

L5 Medial Branch


- Insert the switching stick, cannula, and endoscope
- Transect the L5 medial branch
- Denude the bone at the junction of the S1 SAP and sacral ala

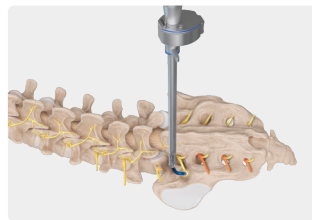

3

S1 Lateral Branch


- Translate the endoscope laterally and inferiorly
- Transect the S1 lateral branch just lateral to the S1 foramen
- Fully denude the underlying bone

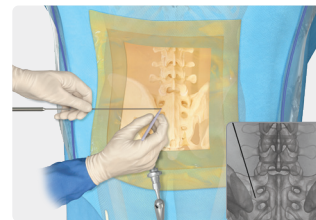

4

Targeting the S2 and S3 Lateral Branches


- Use an orthogonal AP view of sacrum
- Place the needle lateral to the S1 and S2 foramina
- Make an incision for the cannula

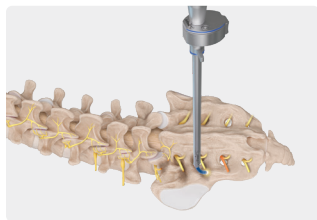

5

S2 Lateral Branch

- Insert the switching stick, cannula, and endoscope
- Transect the S2 lateral branch just lateral to the S2 foramen
- Fully denude the underlying bone

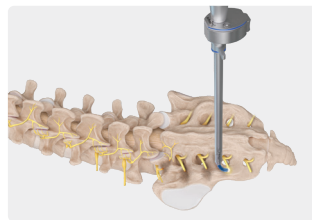

6

S3 Lateral Branch

- Translate the endoscope inferiorly and laterally
- Transect the S3 lateral branch just lateral to the S3 foramen
- Fully denude the underlying bone