



Synergy^{ID} Matrix Secondary Capture Guide

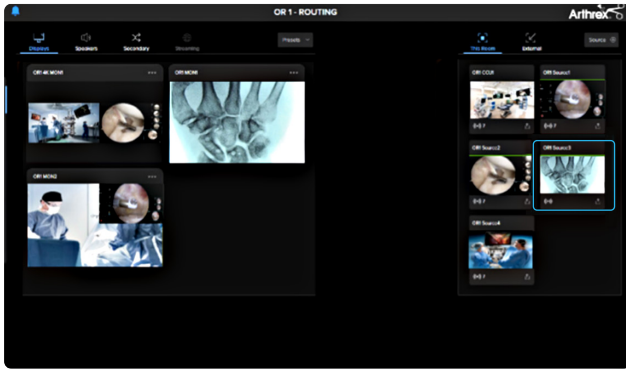


1

From the Synergy Matrix touchscreen controller, select the video source you wish to capture (right).

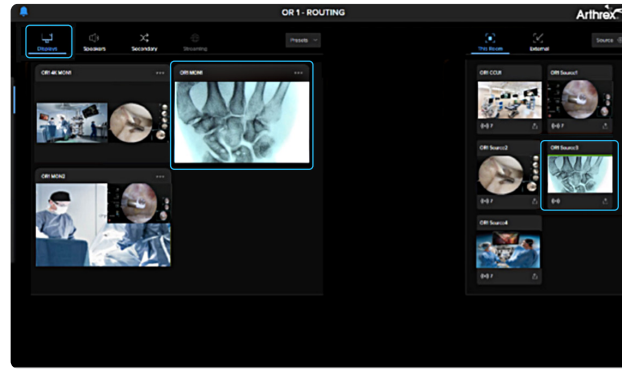


2

On the Display tab, route the video source (right) to your desired display (left).

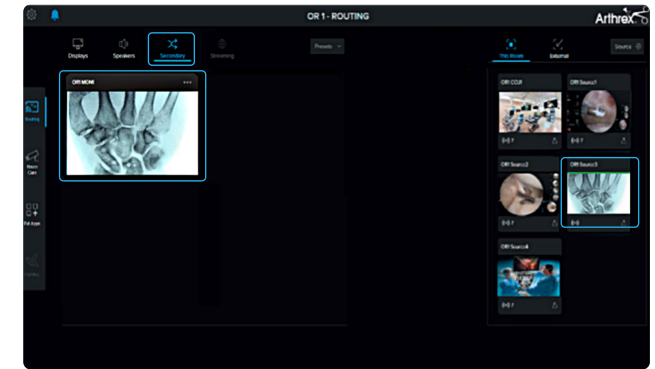


3

Next, tap the Secondary tab. Select the video source you wish to capture (right) and route to the secondary capture destination (left).

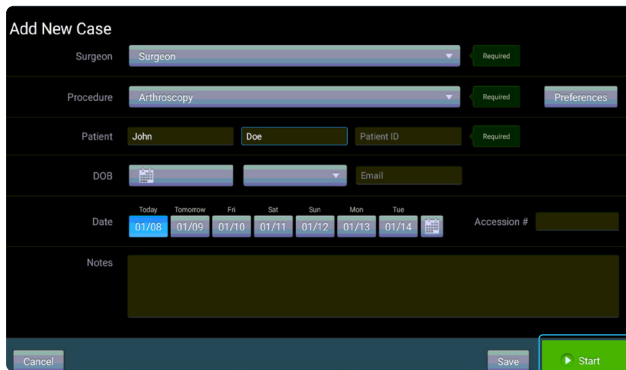


4

On the Synergy^P tablet, enter the patient's information. Once the case is confirmed, press Start to begin.

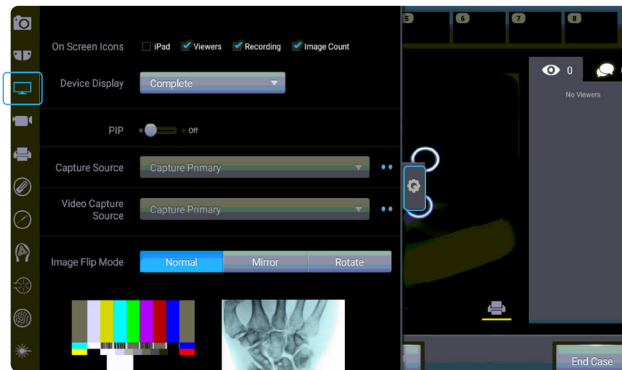


5

From the live case screen, tap the gear icon to access all settings. Select the monitor icon to enter the secondary options menu.

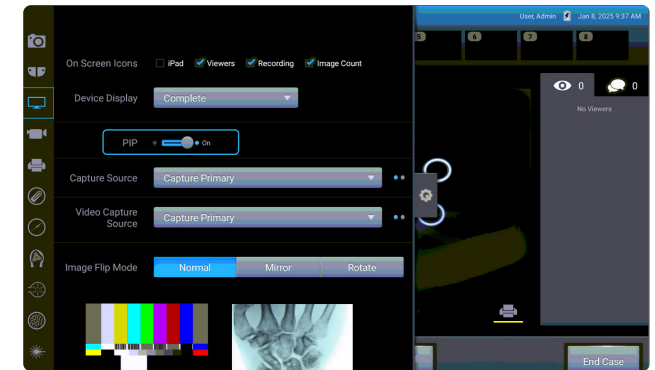


6

Select the PIP slider button to enable the Capture Source drop-down menu.

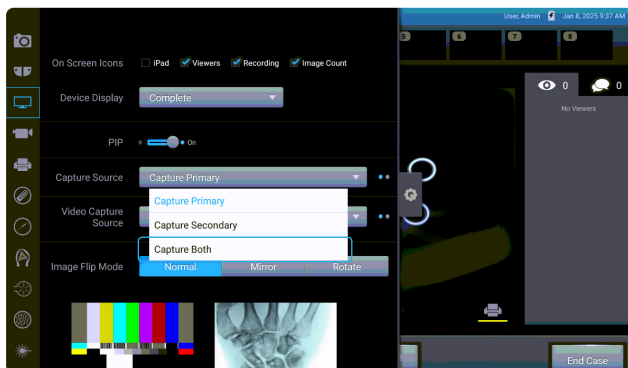

7

From the Capture Source drop-down menu, select Capture Both.

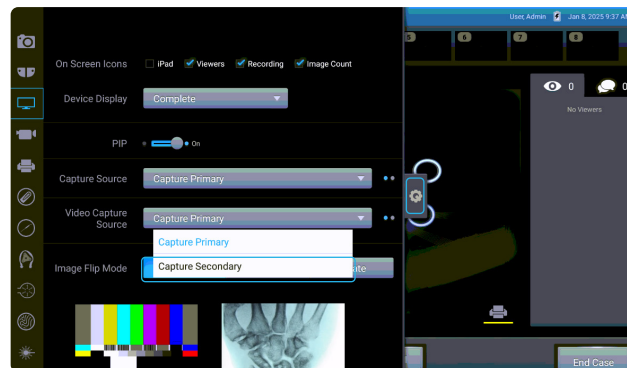


8

From the Video Capture Source drop-down menu, select Capture Secondary if desired. Then tap the gear icon to exit all settings.




9

Tap the camera icon or video icon to capture secondary images or videos.

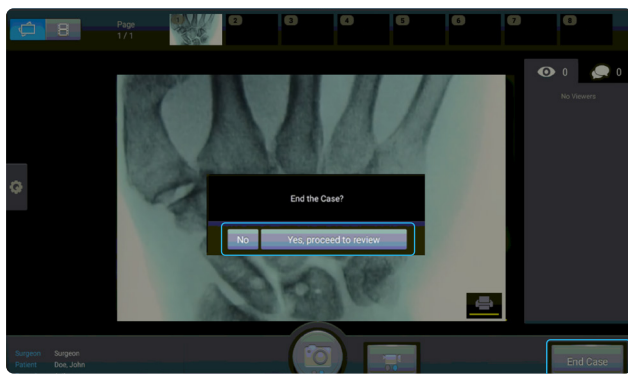

10

When you are ready to end the case, press the gray End Case button at the bottom-right corner and confirm your selection.

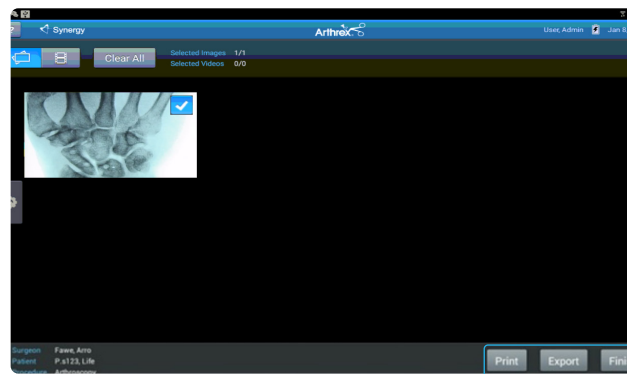

11

Before exiting your case, you will have the option to select your videos or images for print and/or export to the destination of your choice.