

# AlloSync™ Expand

## Hydration Step Guide



1 Peel open packaging and remove product syringe.



2 Remove clear protective cap.



3 Draw hydration fluid and connect to product syringe. NOTE: Use sterile male luer syringe.



4 In vertical position, fully press and release on hydrating syringe plunger 20 times for complete hydration.



5 Unscrew white cap.



6 Extrude the implant material from the syringe prior to implantation.

Product Size	Hydration Volume (mL)
1.0 cc	0.8
2.5 cc	2
5.0 cc	4
10 cc	8

Refer to Instructions for Use for indications and important safety information.

[www.arthrex.com](http://www.arthrex.com)

© 2020 Arthrex, Inc. All rights reserved. LI1-000019-en-US\_A

