

ARTHREX VIDEO CART VERSION II PARTS




MONITOR ARM MOUNT
(002317-00)

CONTENTS


Page Description

2. Video Cart Domestic Views (002317-00)
3. Video Cart Domestic Views (002317-00)
4. Drawer Assembly (002319-00)
5. Power Strip Domestic (304810-00)
6. Side Panel Left (3-001754-00)
7. Side Panel Right (3-001755-00)
8. Rear Handle (304911-01)
9. Hardware Kit, Hinges (305304-00)
10. Cord Wrap Knob (506084-01)
11. Front Cover (304937-00)
12. Patch Panel Assembly (305102-00)
13. IV Bag Hanger, LH *as facing cart* (002352-00)
14. Adjustable Shelf (304909-00)
15. CO2 Tank Assembly Kit (002321-00)
16. Pull-Out Pump Shelf; Version II (002318-00)
17. Front Handle (Left/Right) (304955-01)
18. Foot Pedal Holder Assembly (305344-00)
19. Casters, Locking (703548-15), Kit of 4 (002493-00)
20. Power Cables, 120V~ (002363-00)
21. Boom Arm With Logo (3-001549-00) Hardware kit (3-001548-00)
22. Home Position Bracket (304871-00)
23. Dual Yoke Valve (002433-00), Kit (305590-00)
24. Drawer w/o Shelf (303914-00)
25. Transformer Domestic, 120 VAC (002305-00)
26. Monitor Arm Handle Kit (002340-00)
27. Logo (505050-00)/ Logo, Front (505051-00)
28. Touch-up Paint Gray (702465-20)/ Touch-up Paint Nickel (702481-20)
29. Pivot 3 Assembly (304803-03)
30. Switch Replacement Kit (305129-00)
31. Monitor Arm Conversion Kit (305468-00)
32. Secondary Arm Lower Assembly (3-001546-00)
33. 14-3 SJT Cord Set 5-15P to C-13 HG (15) (702078-30)
34. Pages 34-37 Remote Switch Replacement Instructions


**VIDEO CART
(002317-00)**

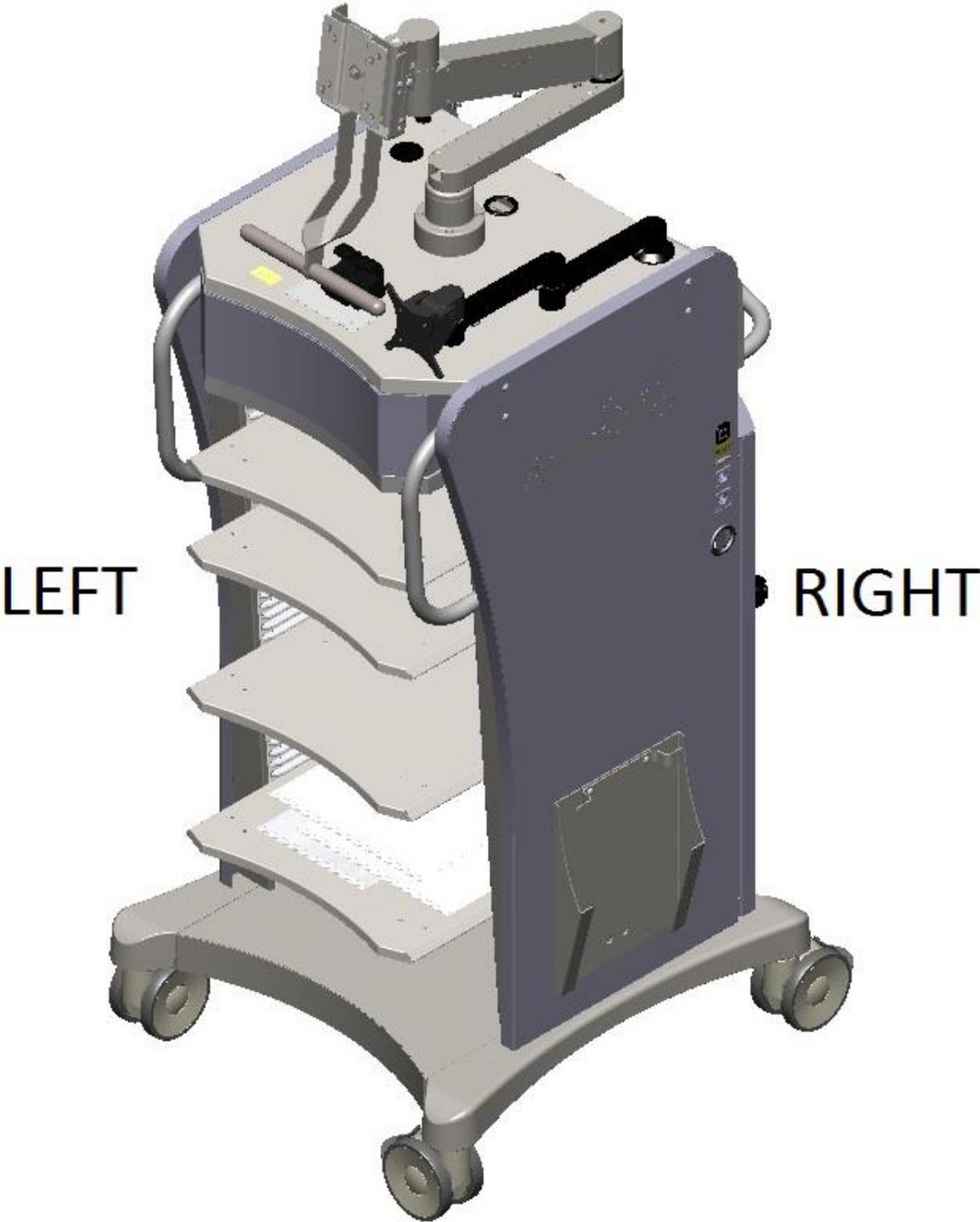


**VIDEO CART
(002317-00)**

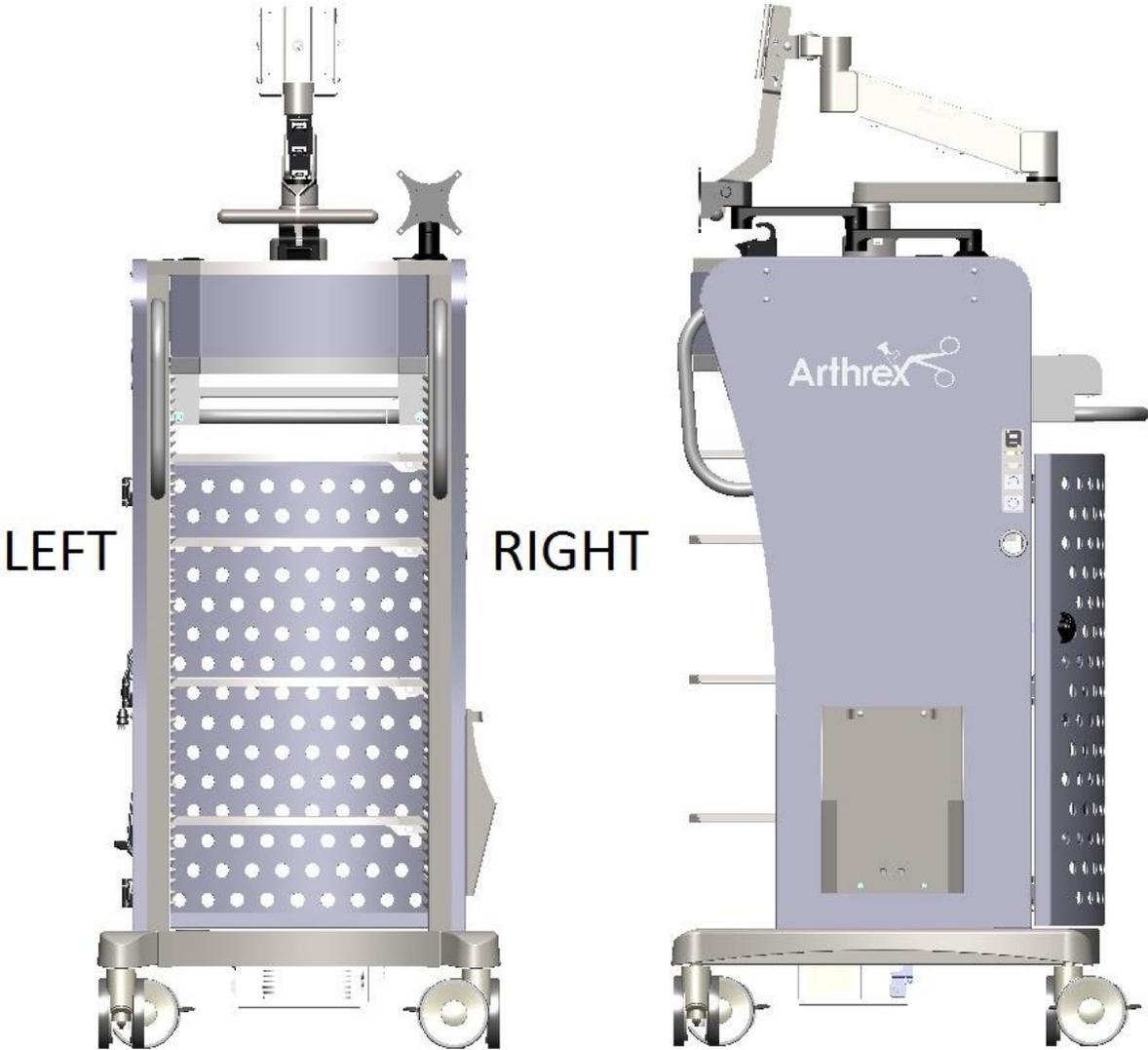


DRAWER ASSEMBLY
(002319-00)

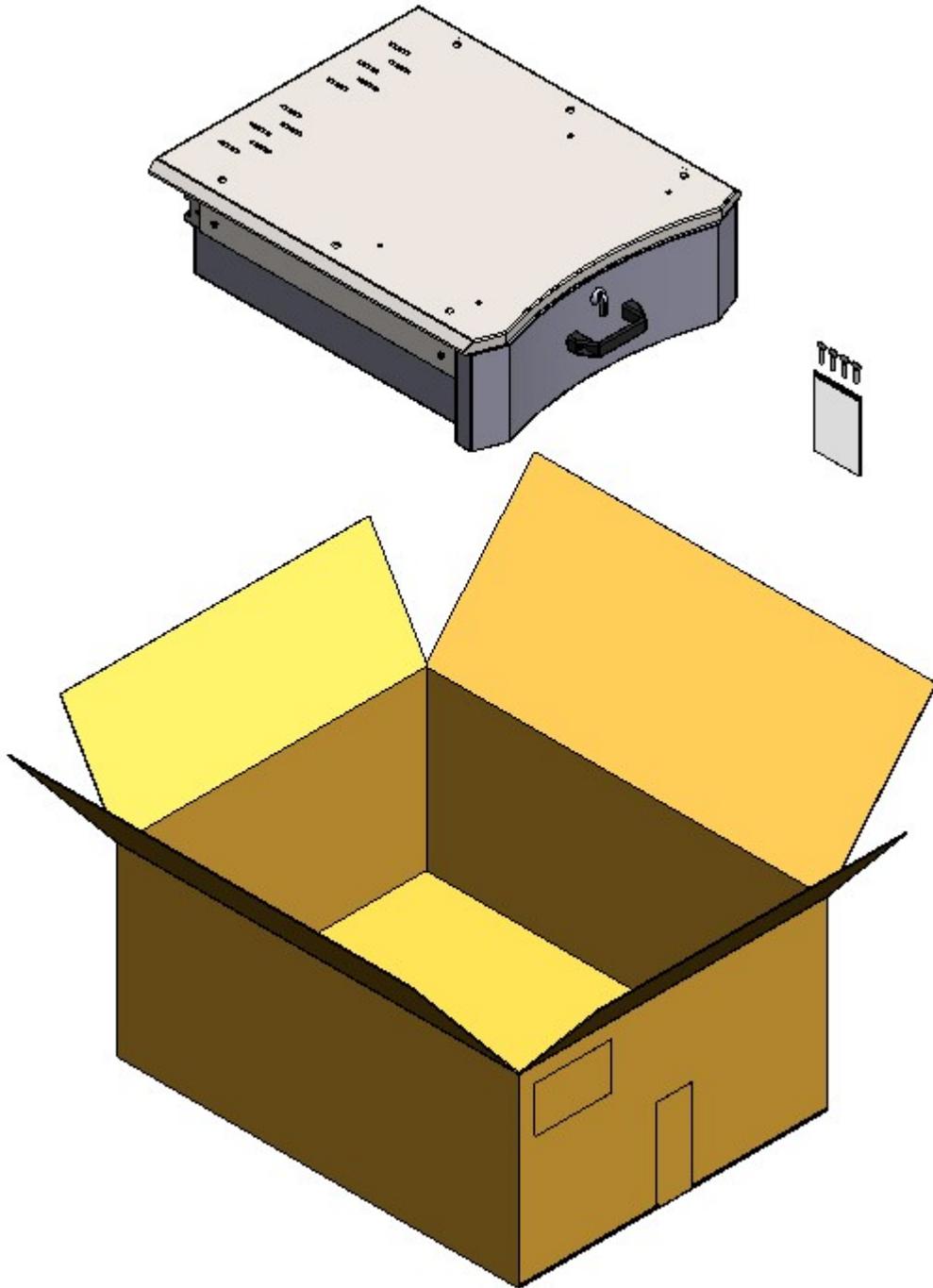


POWER STRIP, DOMESTIC
(304810-00)

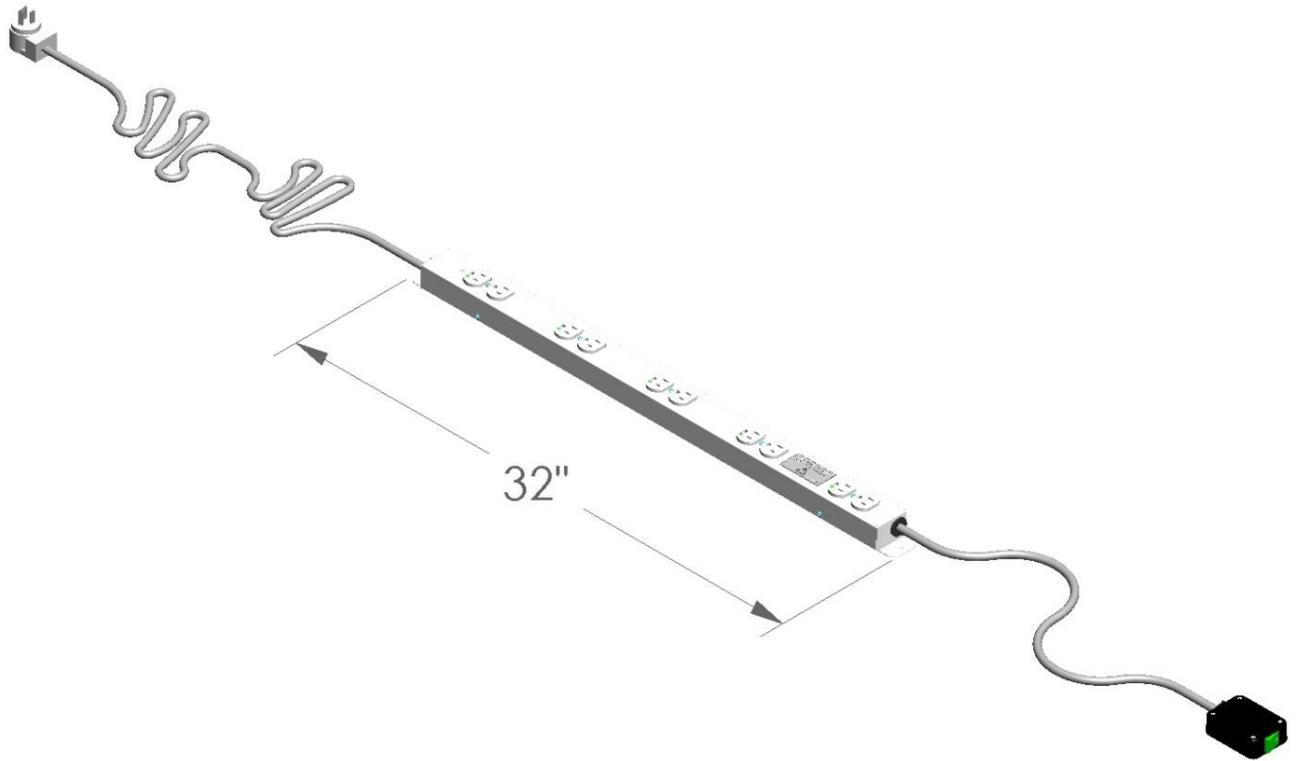


SIDE PANEL LEFT WITH LOGO
(3-001754-00)

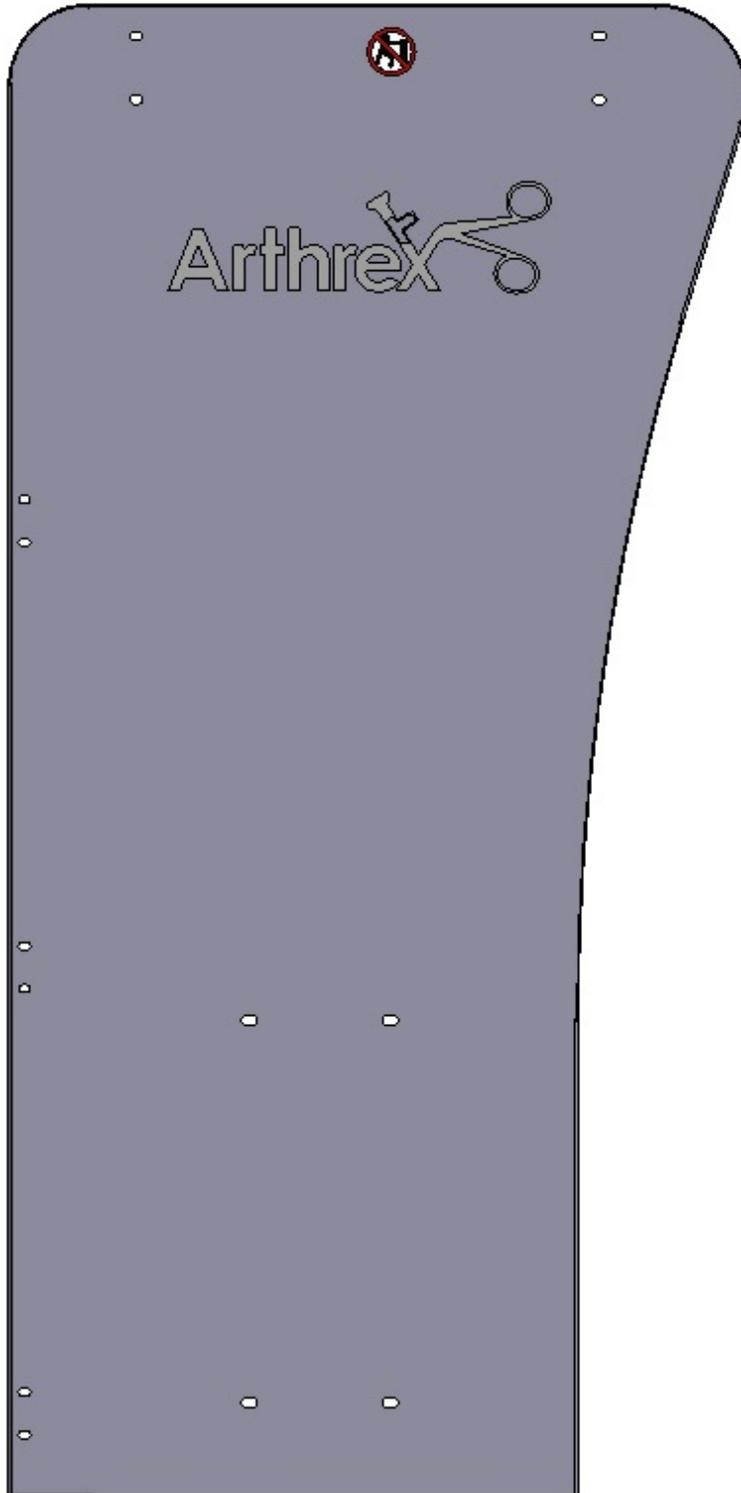



REAR HANDLE
(304911-01)

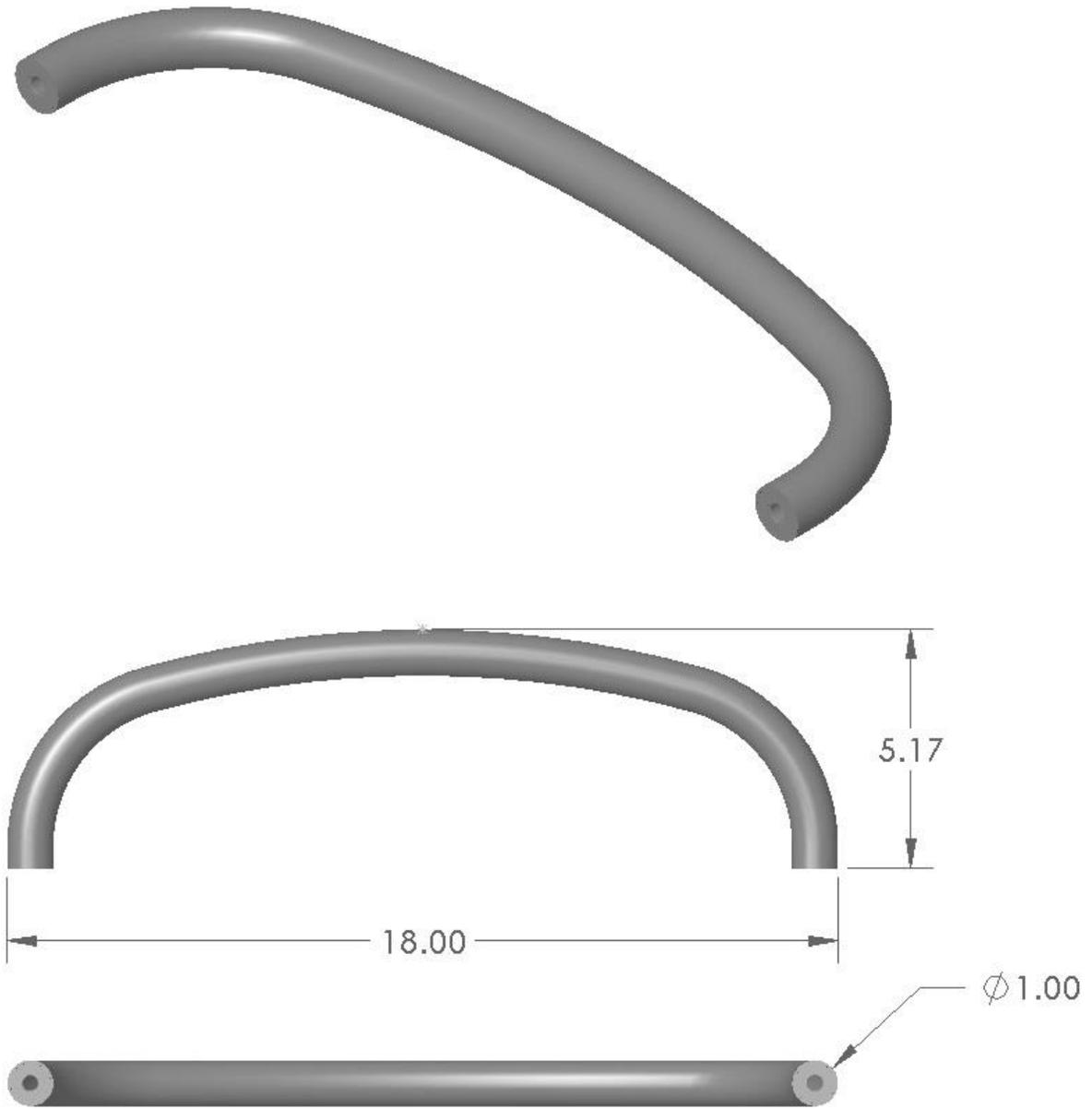


HARDWARE KIT, DOOR HINGES
(305304-00)

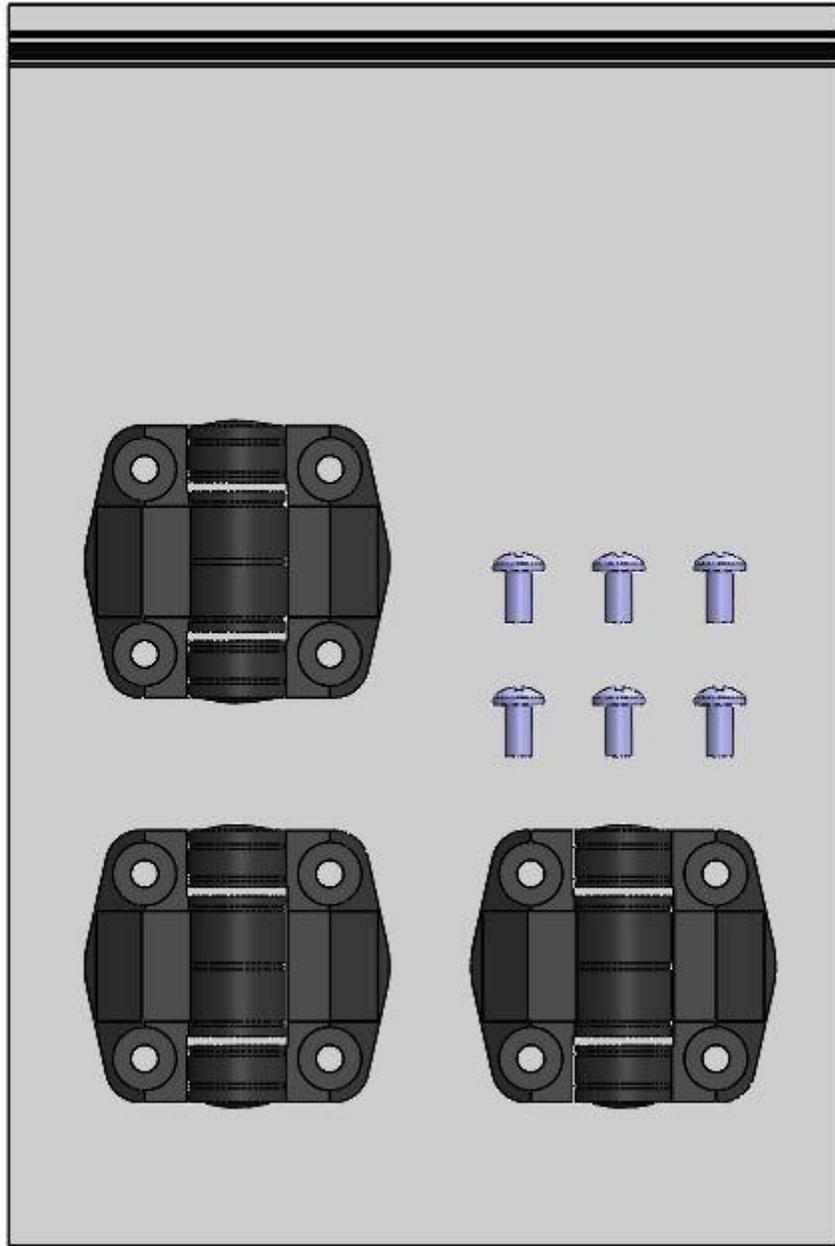

CORD WRAP
(506084-01)

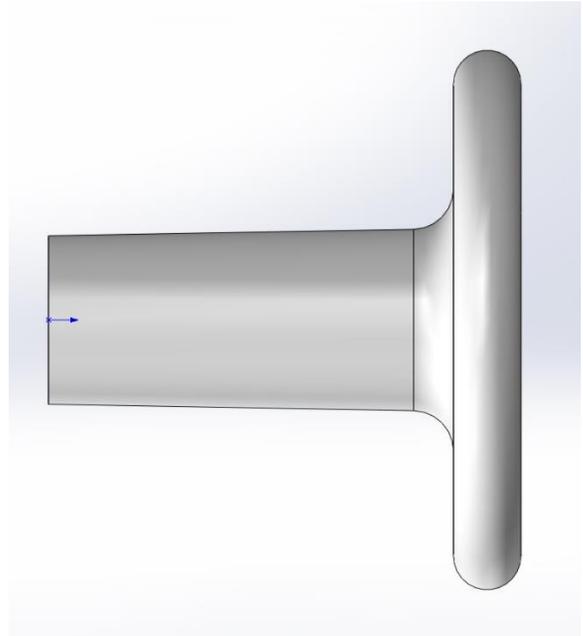
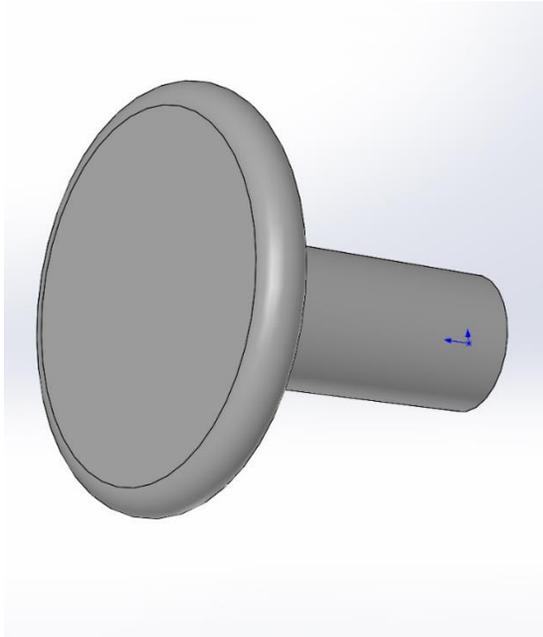


FRONT COVER
(304937-00)

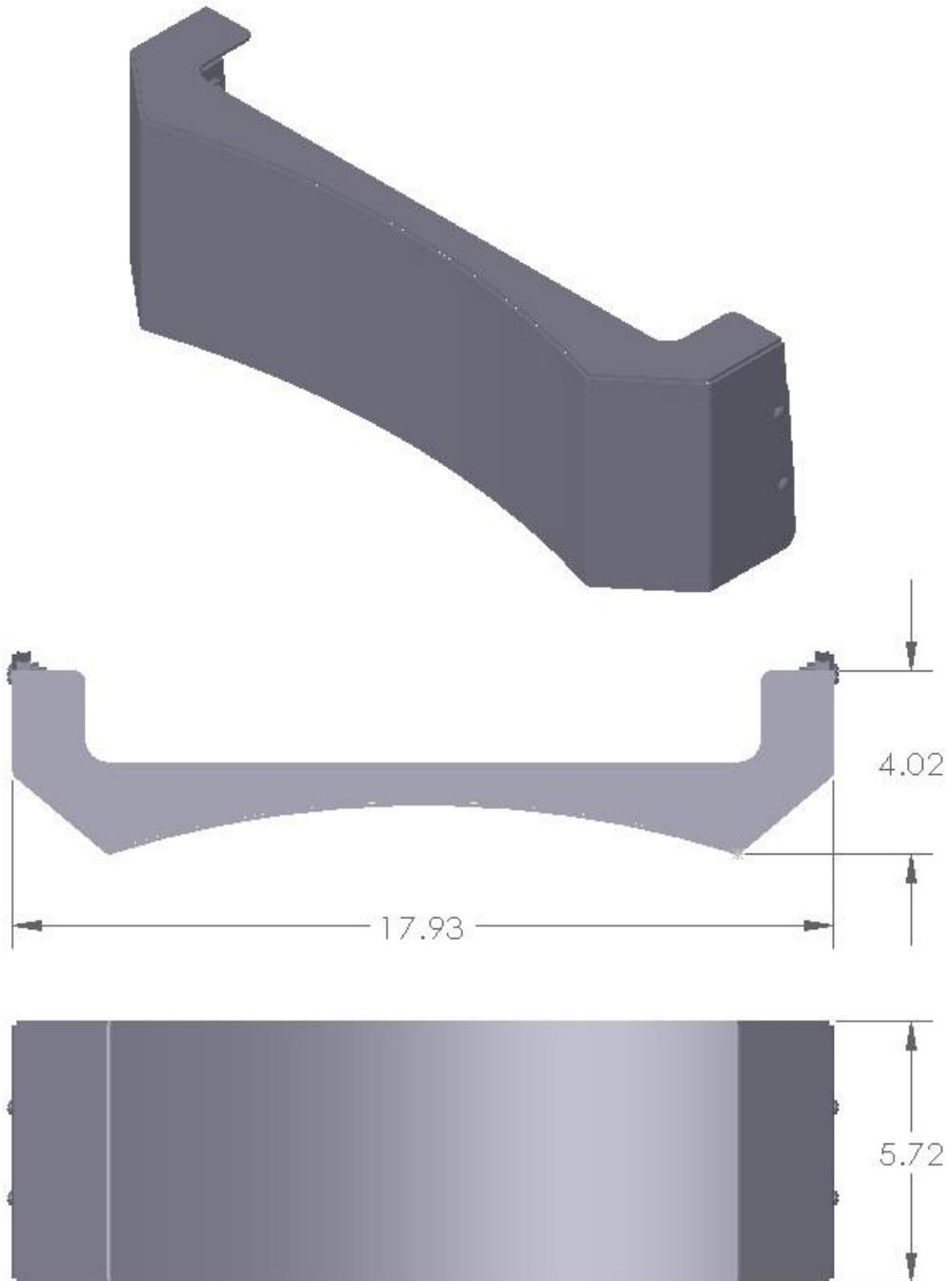



PATCH PANEL ASSEMBLY

(305102-00)

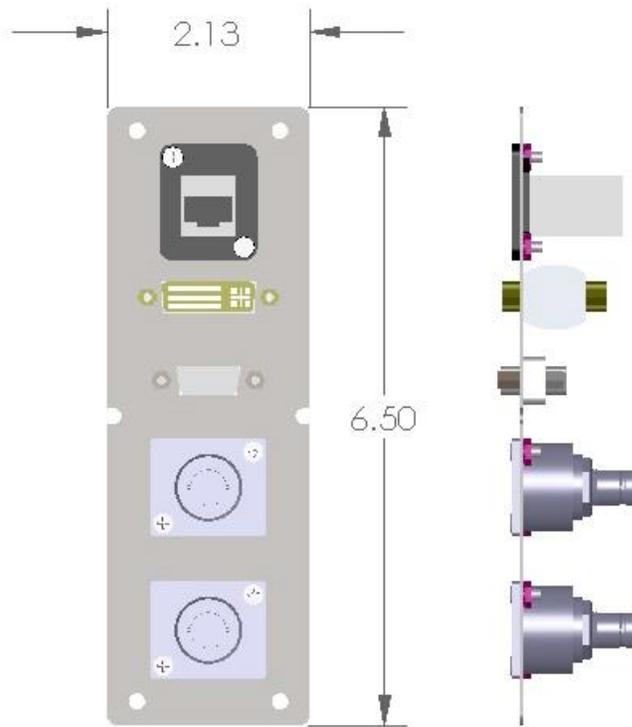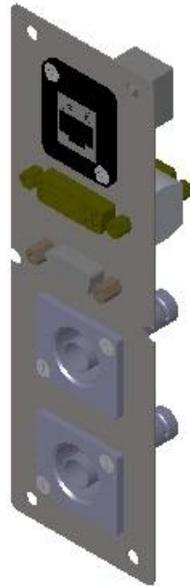





IV BAG HANGER, LH

(002352-00)

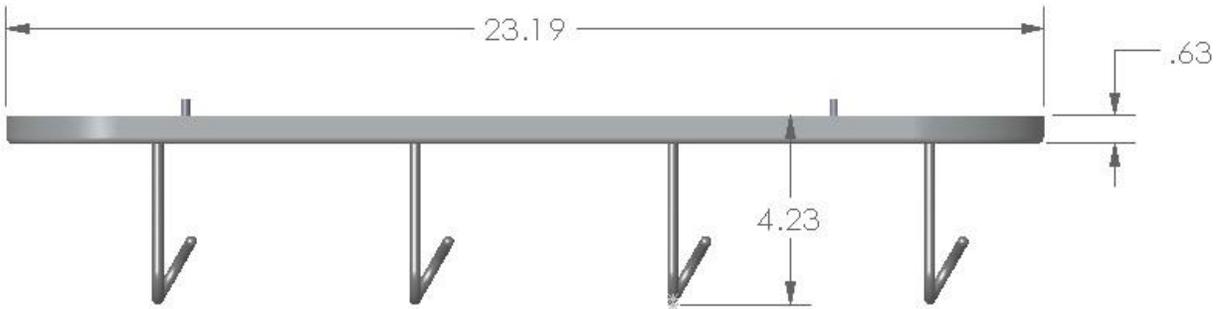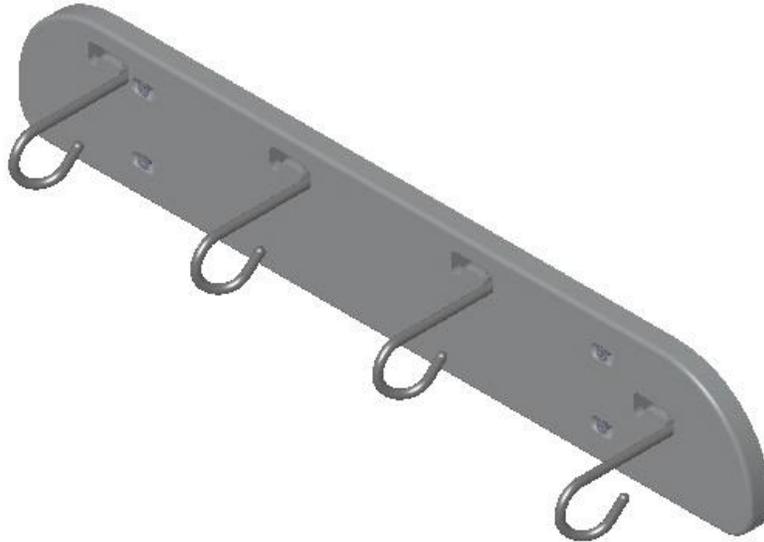


NOTE: RH IV BAG HANGER IS NOT AVAILABLE DUE TO LOCATION OF ELECTRICAL CONNECTIONS.


ADJUSTABLE SHELF (304909-00)

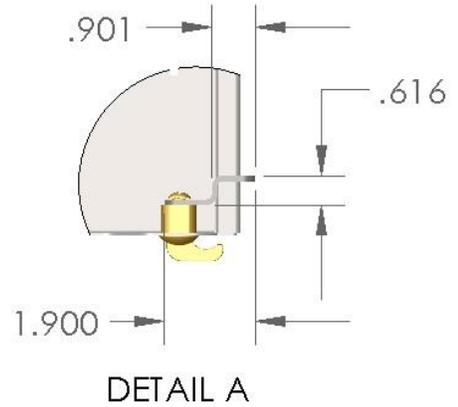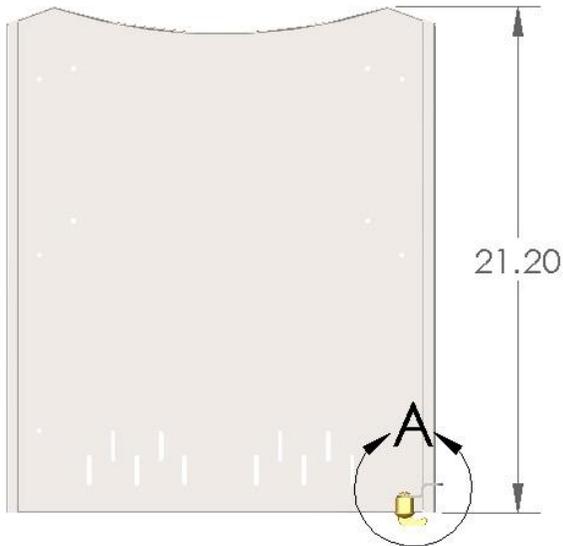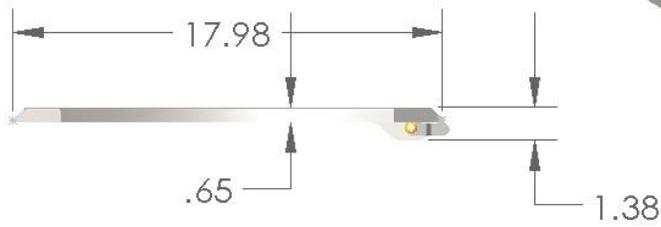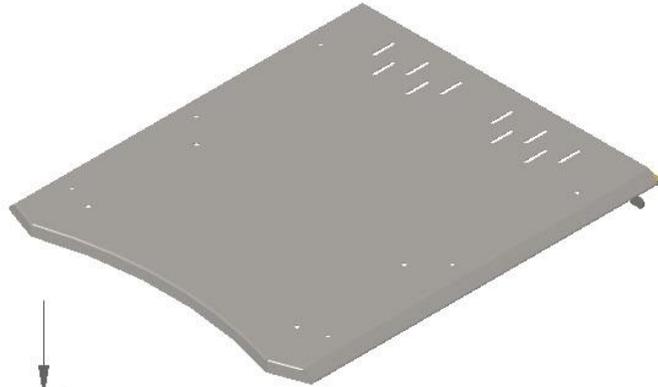


**CO2 TANK ASSEMBLY KIT
(002321-00)**

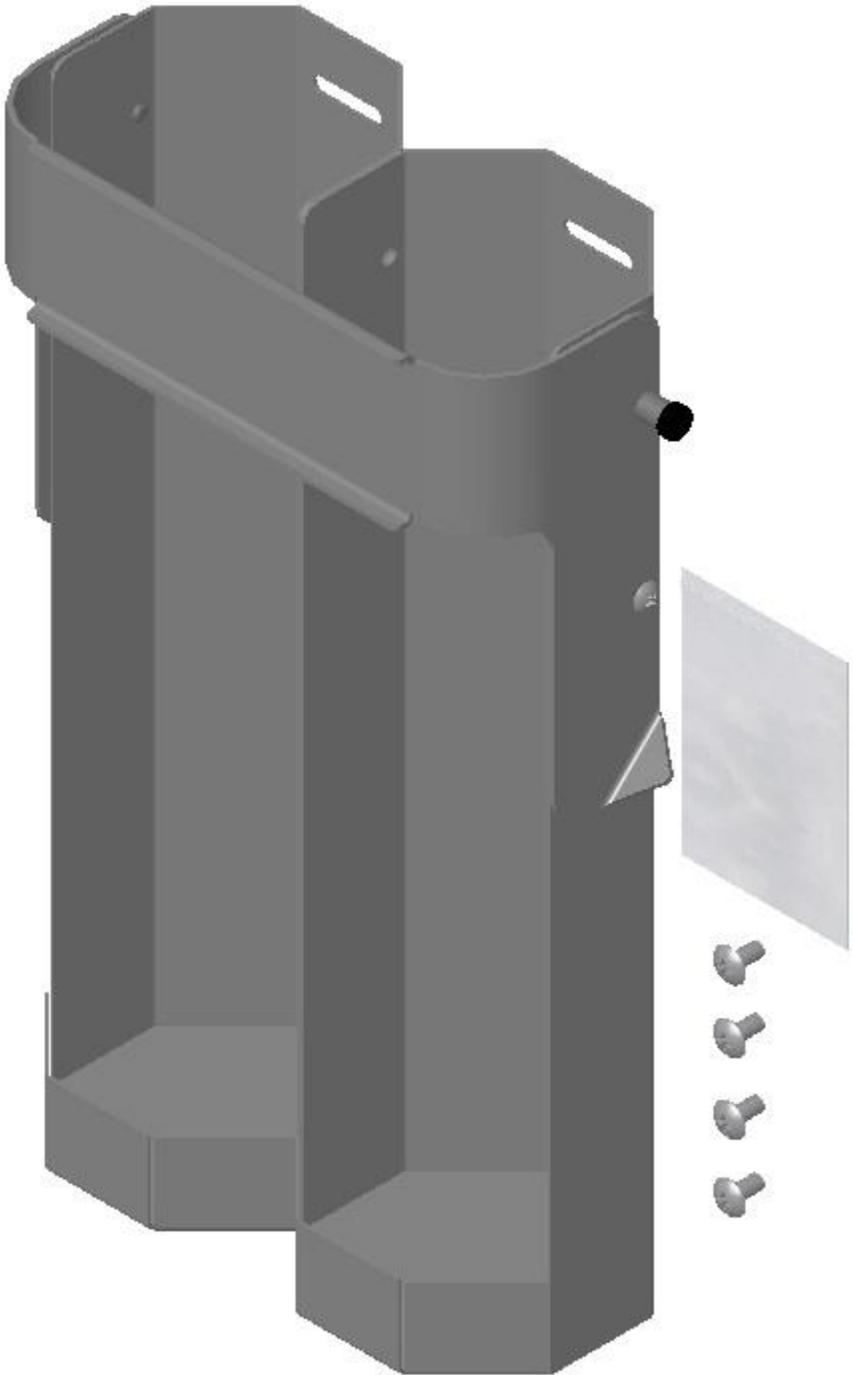


PULL-OUT PUMP SHELF (002318-00)

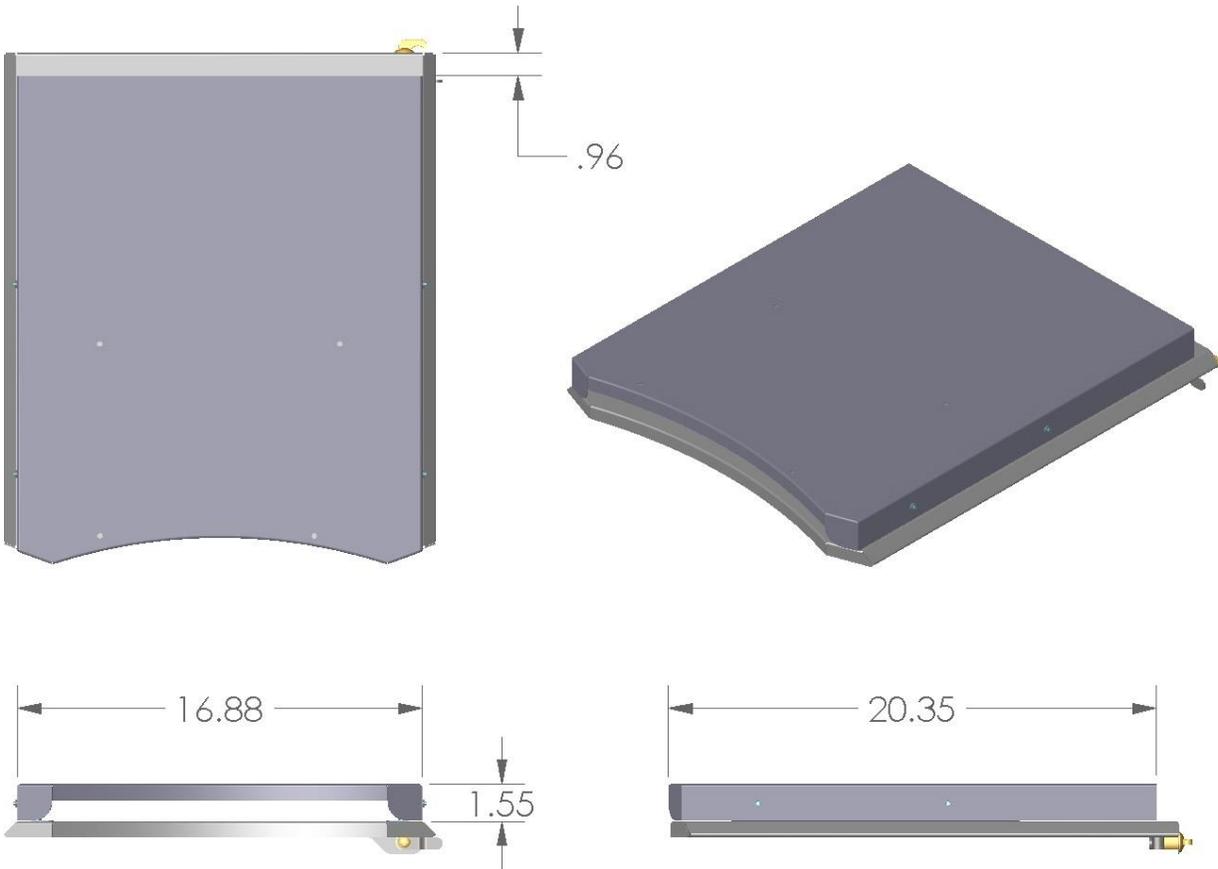


FRONT HANDLE
(304955-01)

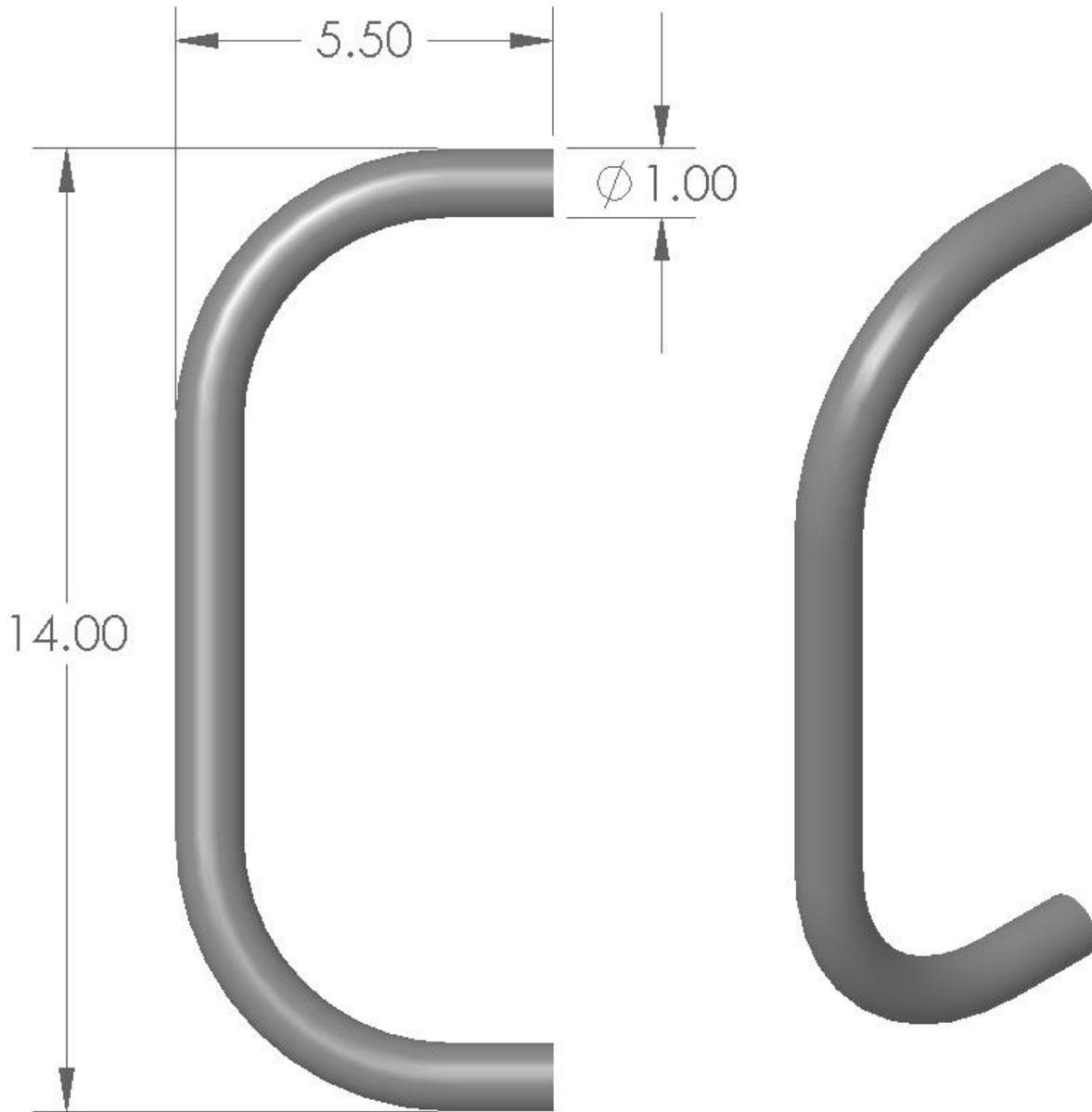

FOOT PEDAL HOLDER ASSEMBLY
(305344-00)

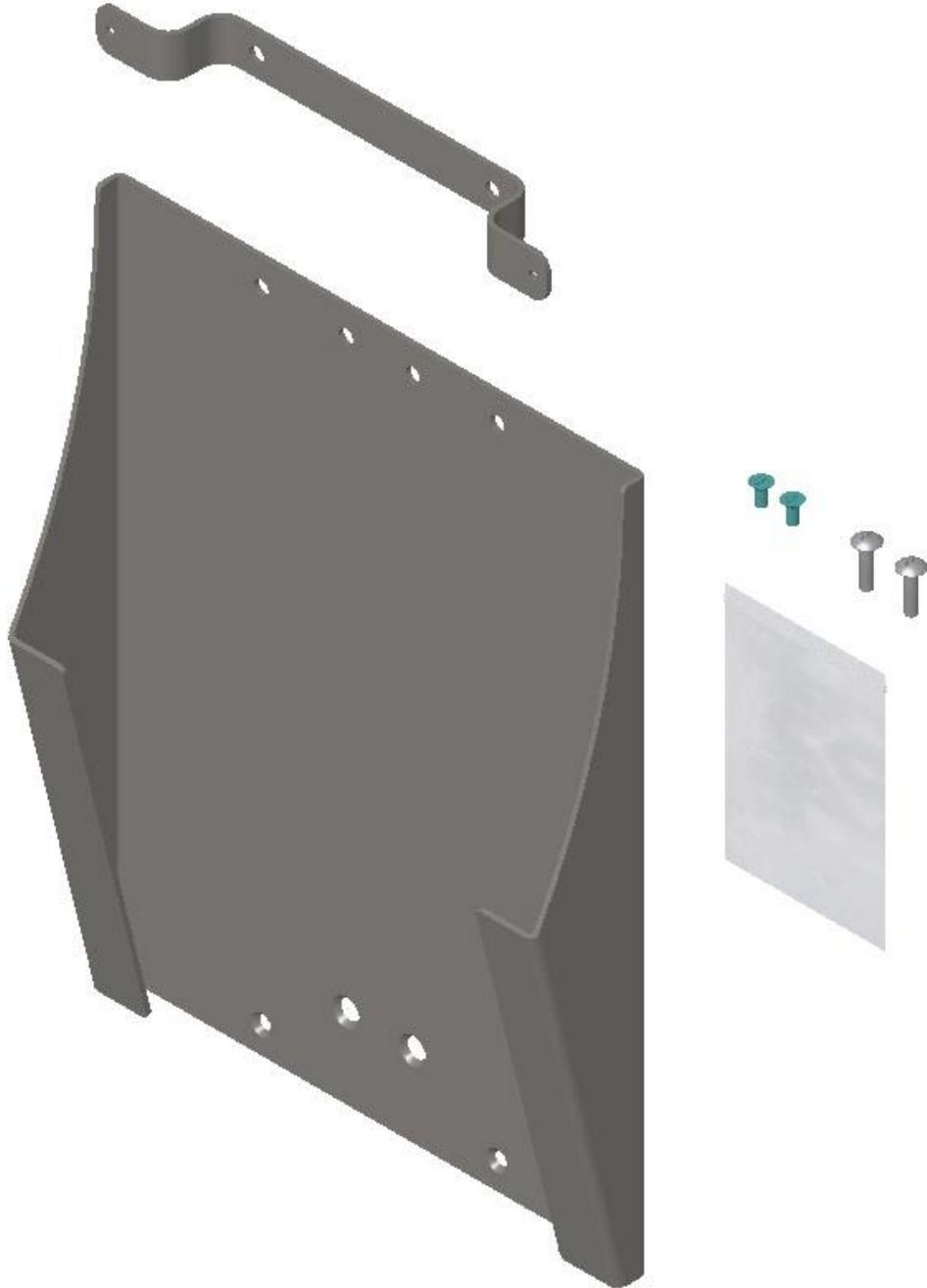

CASTER, LOCKING
(703548-15)

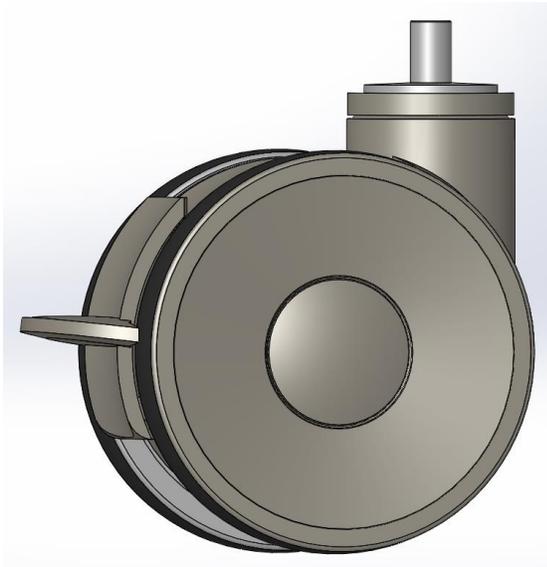

NOTE: THIS CASTER UTILIZES A HARDENED BOLT FOR ATTACHMENT TO CART. CASTERS WITH BOLT CAN NOT BE USED IN CONJUNCTION WITH EXISTING CASTERS. ANY REPLACEMENT REQUIRES ENTIRE SET OF FOUR CASTERS REPLACED.

CASTER KIT (4)
(002493-00)


Note: Loctite primer (7471) and red thread locker (2760) are not included due to shelf life issues.





POWER CABLES, 120V~
(002363-00)




BOOM ARM WITH LOGO (3-001549-00)

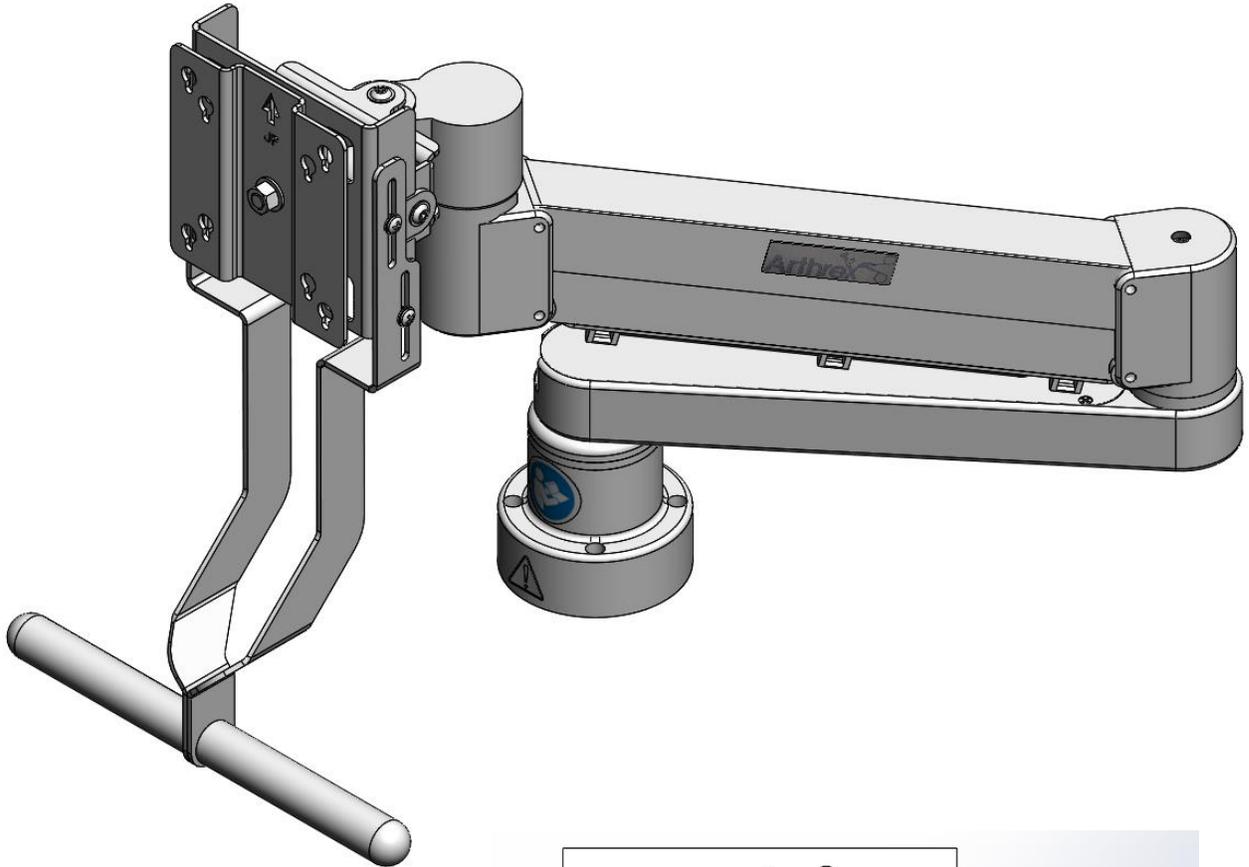

BOOM ARM INSTALLATION KIT (3-001548-00)

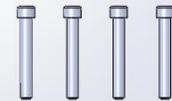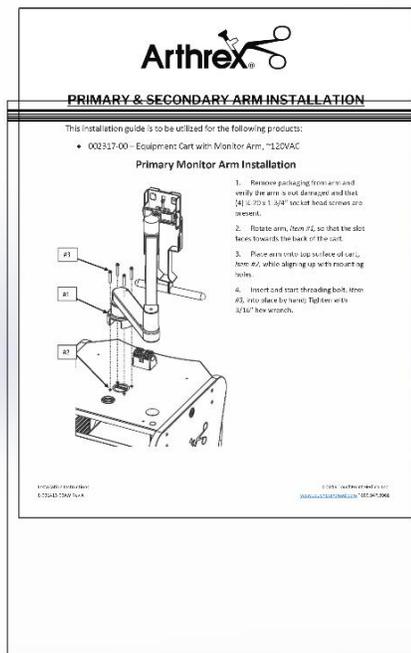


HOME POSITION BRACKET
(304871-00)

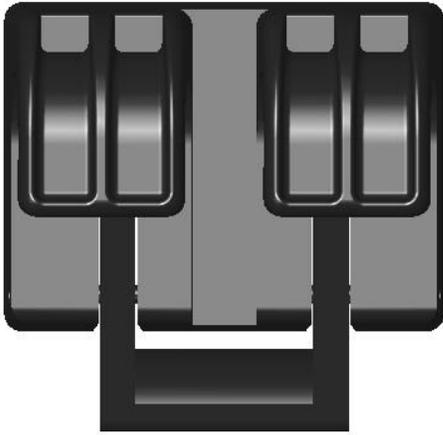


TOP



ISOMETRIC

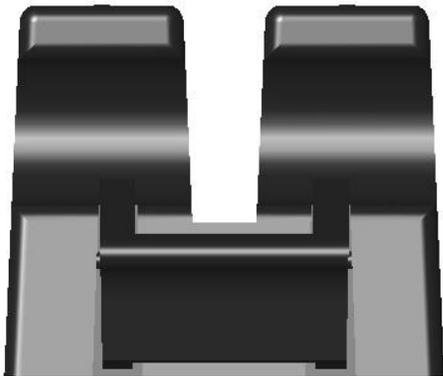


FRONT

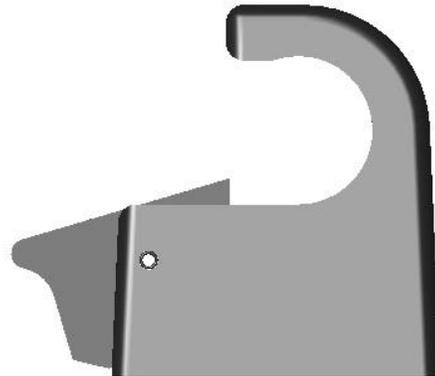


SIDE

For use with Boom Arm





**DUAL YOKE WITH
HARDWARE KIT
(002433-00)**

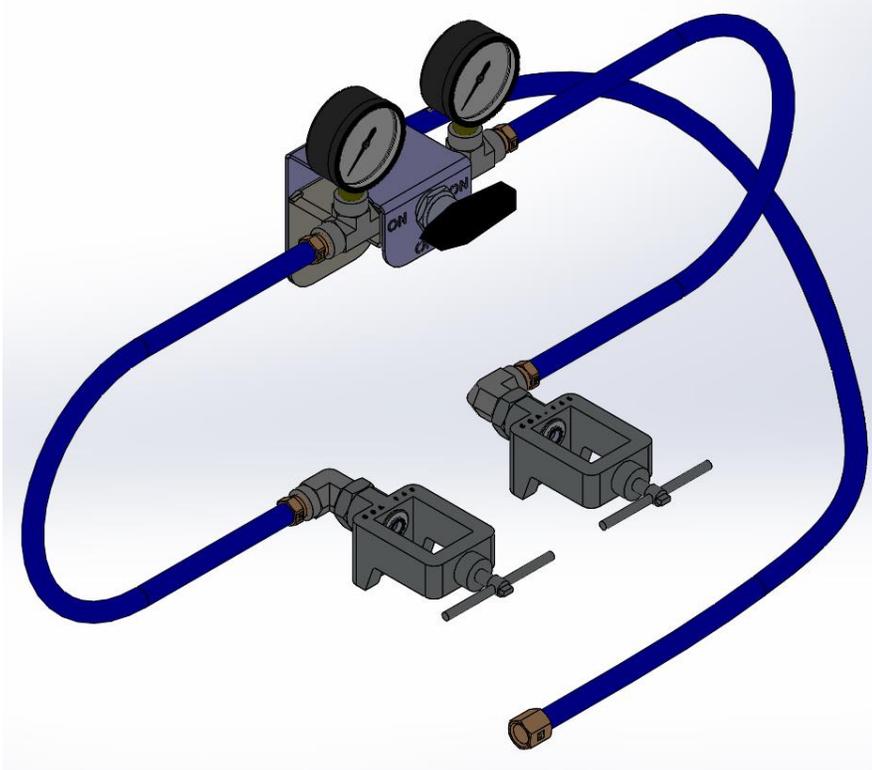


**HARDWARE KIT
DUAL YOKE
(305590-00)**

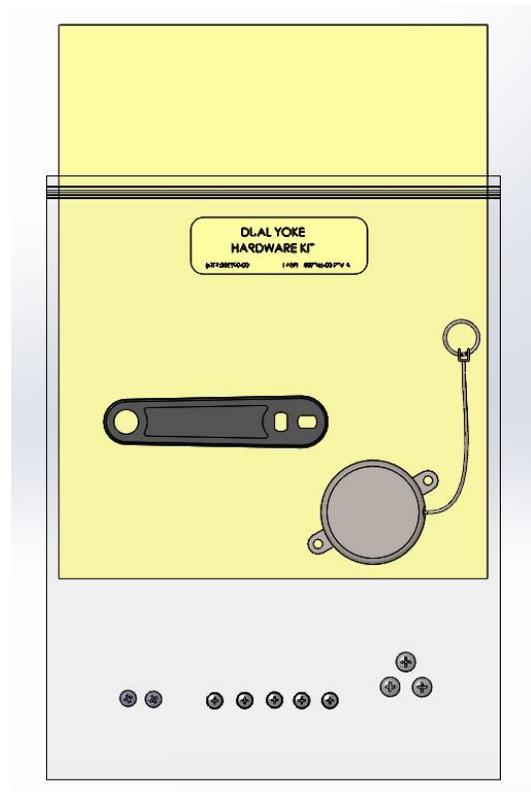


DRAWER W/O SHELF (303914-00)

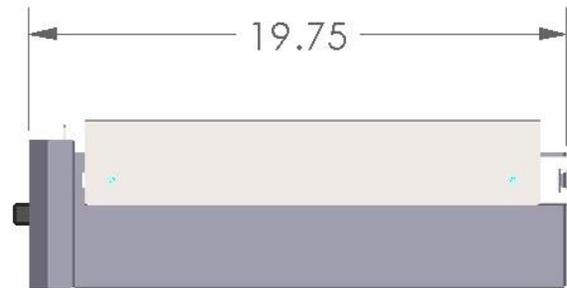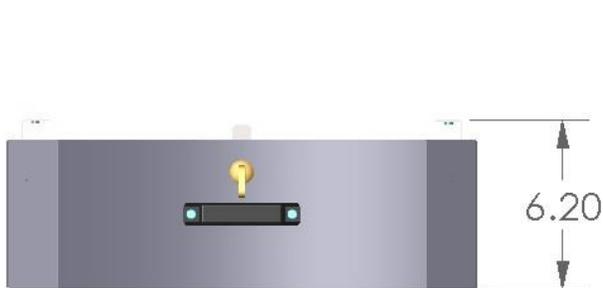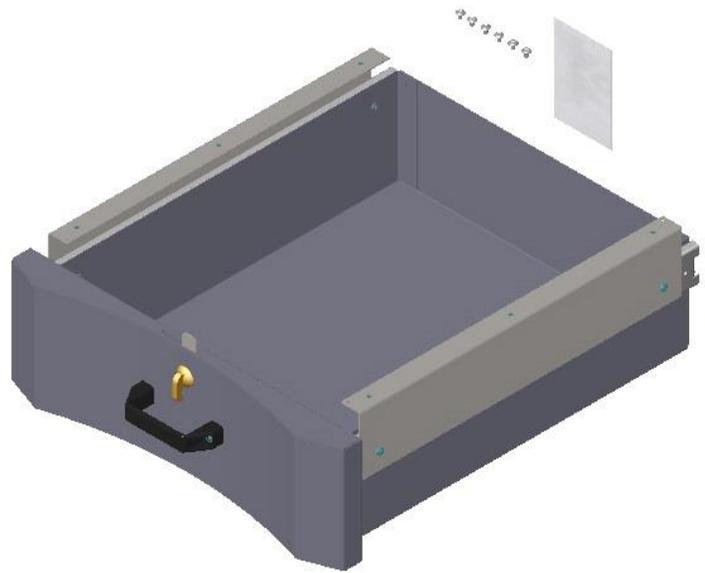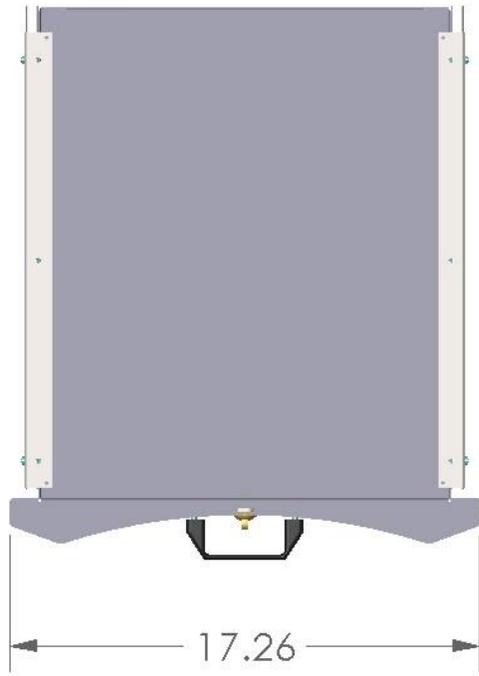

TRANSFORMER DOMESTIC, 120 VAC (002305-00)

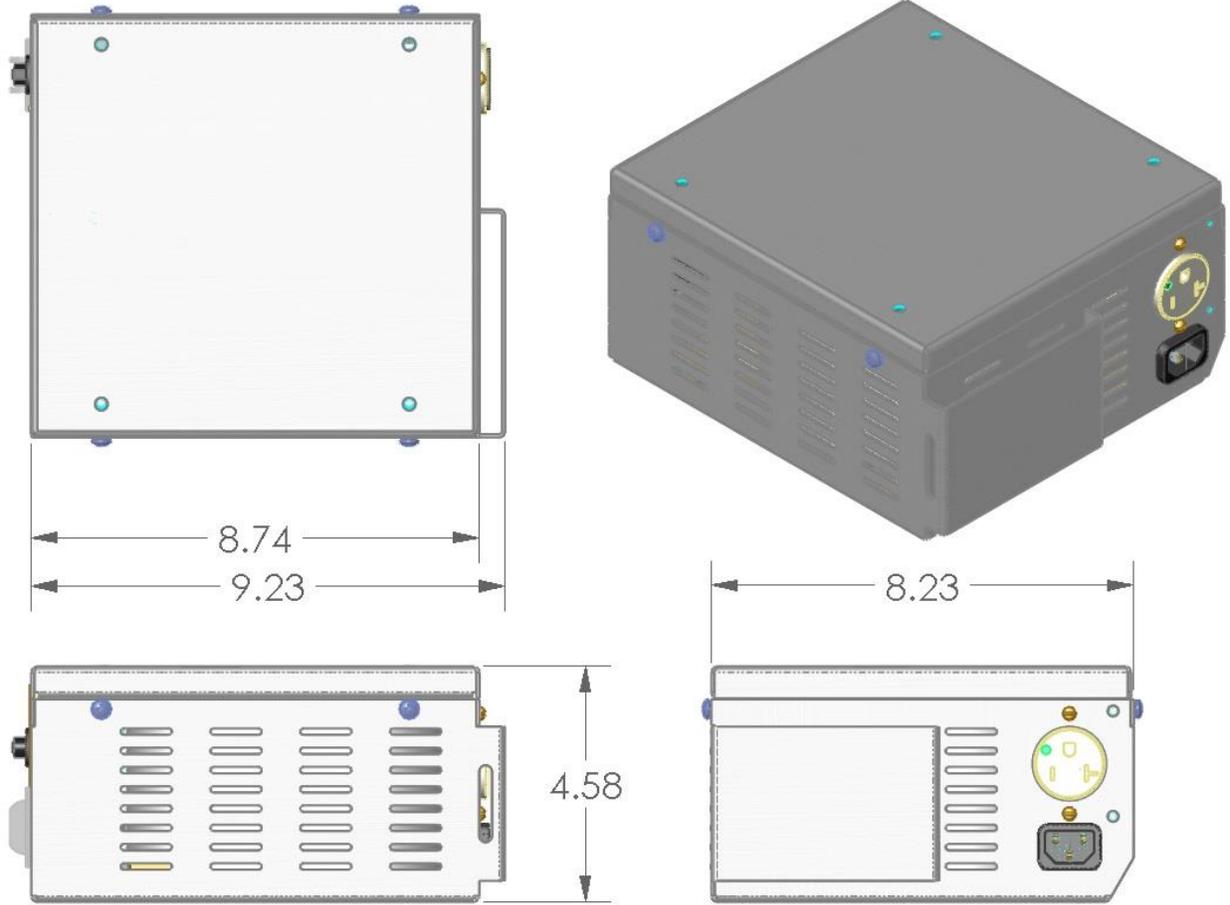

MONITOR ARM HANDLE KIT
(002340-00)

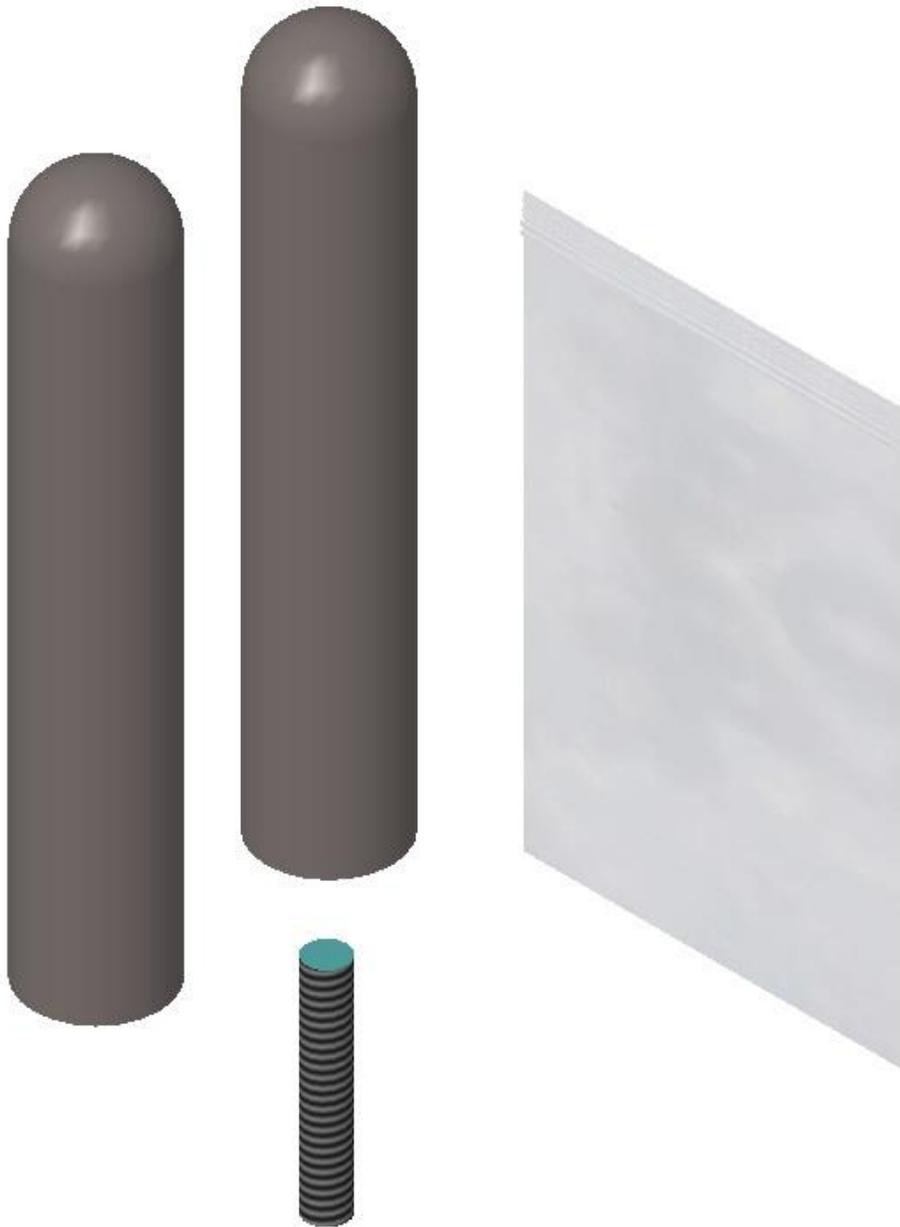

LOGO
(505050-00)




LOGO, FRONT
(505051-00)




TOUCH-UP PAINT GRAY

(702465-20)

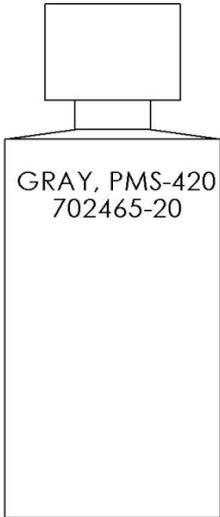


TOUCH-UP PAINT NICKEL



(702481-20)

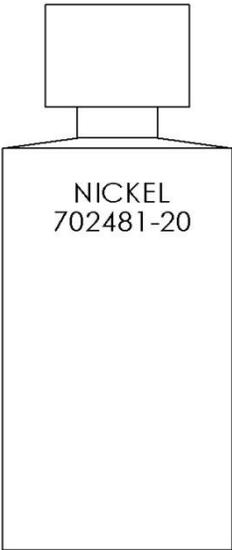


**PIVOT 3
ASSEMBLY
(304803-03)**

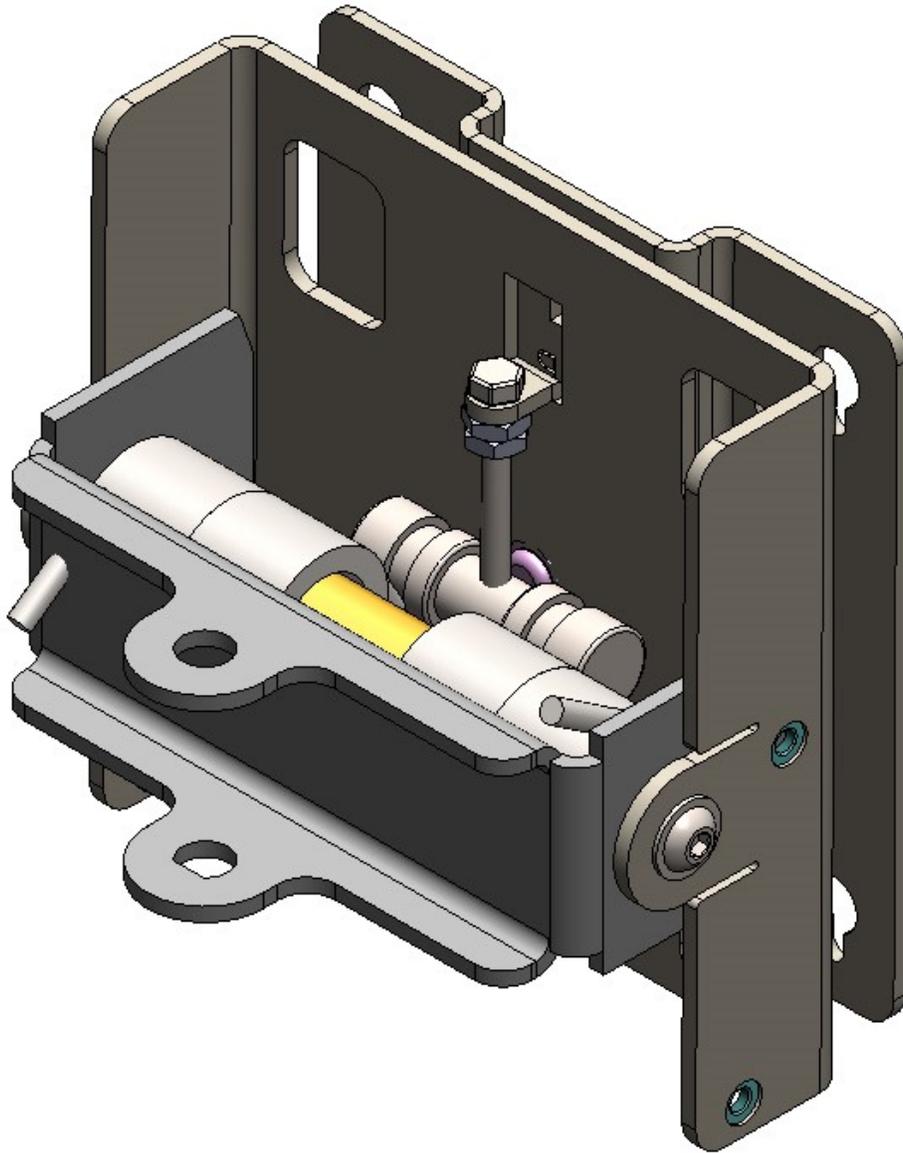


**SWITCH
REPLACEMENT KIT
(305129-00)**

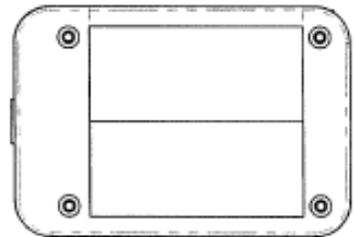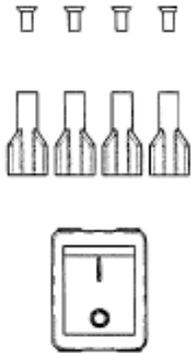

**MONITOR ARM
CONVERSION KIT
(305468-00)**

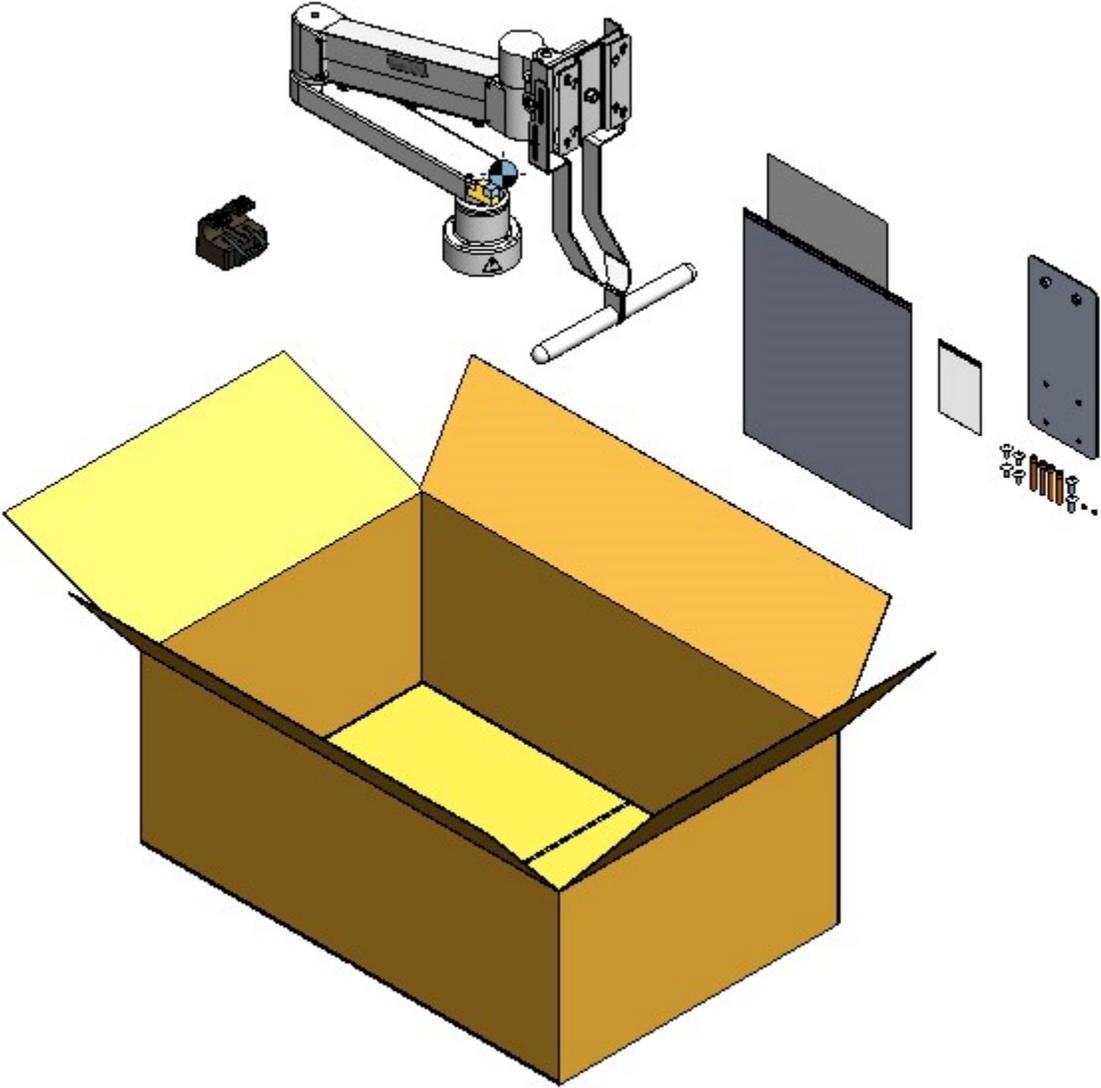

**SECONDARY ARM
LOWER ASSEMBLY
(3-001546-00)**

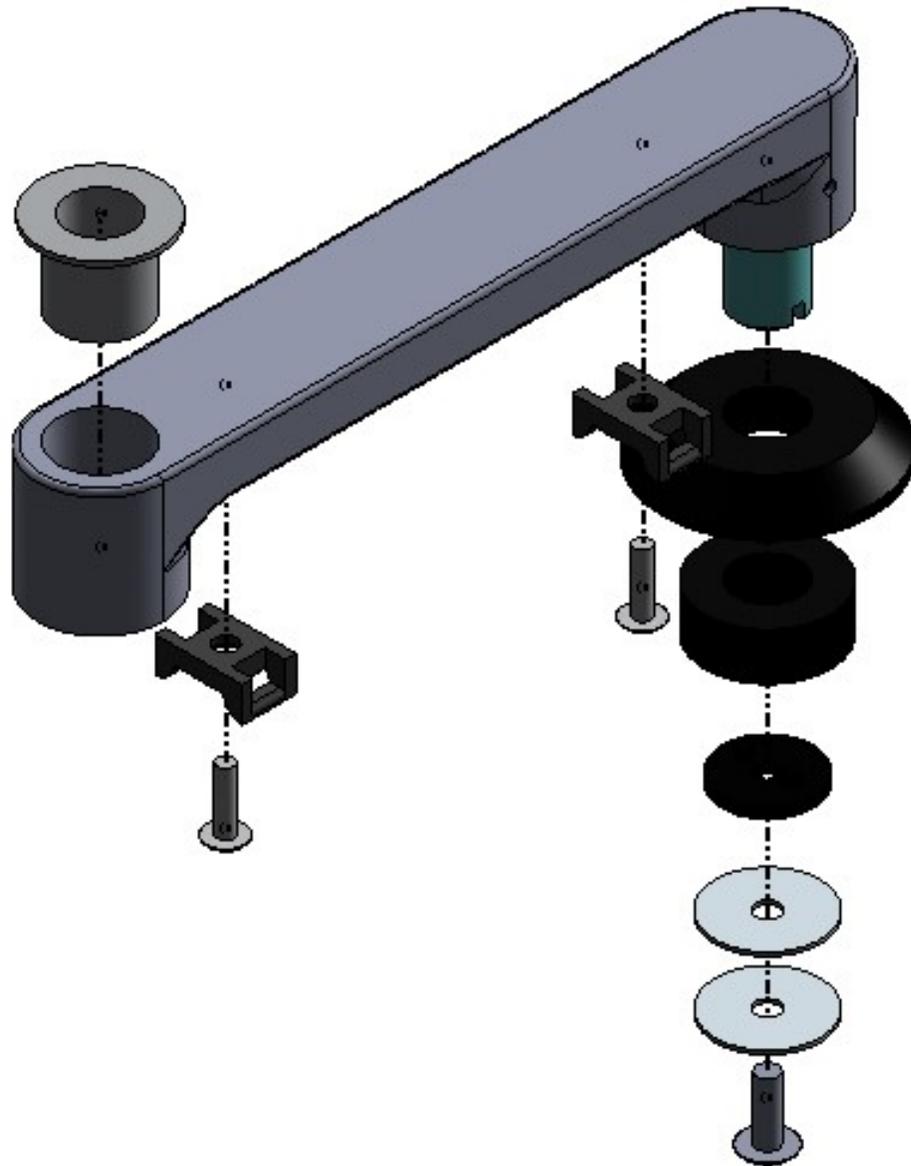

Remote Switch Replacement: TouchPoint Medical Grade Power Strip Arthrex AR-6000 Series Video Tower

Warnings

- Read the following documents completely prior to use. Please read this manual carefully; it is very important to be thoroughly familiar with these instructions.
- Remove all power to the cart. Ensure that all power cords are unplugged.
- Only a qualified Biomed should perform the power switch replacement.

Tool / Hardware Kit


- You should receive a Tool / Hardware Kit that includes the following:

QTY	ITEM DESCRIPTION
1	SWITCH CASE
1	SWITCH CASE COVER W/ DOUBLE SIDED TAPE
1	GREEN ILLUMINATED ON/OFF SWITCH
4	MOLEX QUICK DISCONNECTS
4	4-40 x 1/4 FHPHUC SCREWS
1	MOLEX CRIMPING TOOL
1	#1 PHILIPS HEAD SCREWDRIVER
1	1 1/32 IN NUT DRIVER
1	WIRE STRIPPER
1	PLIERS
1	WIRE CUTTERS
1	STRAIN RELIEF PLIERS
1	RECEPTACLE TESTER

Note: The Tool Hardware Kit is valued at \$750. Upon completion please place ALL materials into the dedicated shipping container and place the prepaid Return Address Label on the outside of the box. The contents will be inspected upon receipt to ensure all tools are included; customer will be invoiced if all contents are not returned.



Remove remote switch box assembly from the cart. The remote switch is secured to the cart with double sided 3M VHB tape.

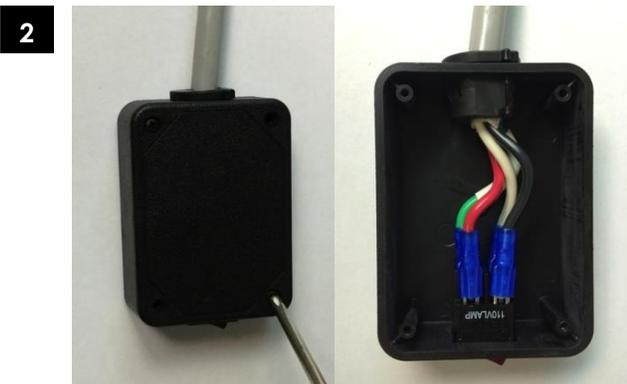


Remove the top cover plate of the plastic enclosure by removing the four corner screws with the enclosed #1 Philips-headscrewdriver.

3



Disconnect the QTY (4) quick disconnects of the switch terminals. Cut the quick disconnects (blue connectors) as shown.
NOTE: The terminals of the replacement (Carling LRGSC Series) switch are larger so the original quick disconnects need to be removed.


4

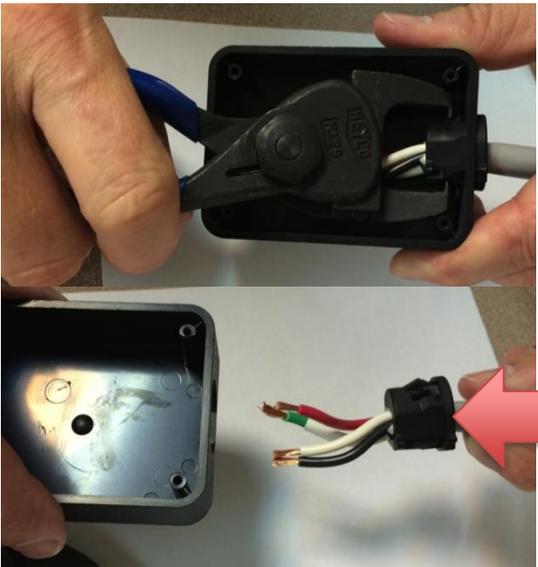

Remove the black strain relief with the enclosed strain relief pliers. Depress the tabs on the strain relief and remove the cable assembly from the plastic housing.

Remove strain relief.

5

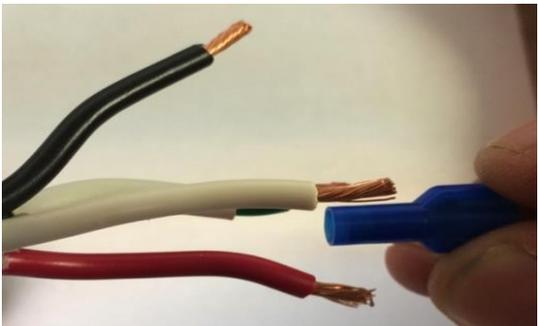


Make sure that there is 1/4 in of exposed wire; strip wire jacket with enclosed wire strippers if necessary.

6




Place the Molex quick disconnects into the enclosed Molex crimping tool as shown.

7

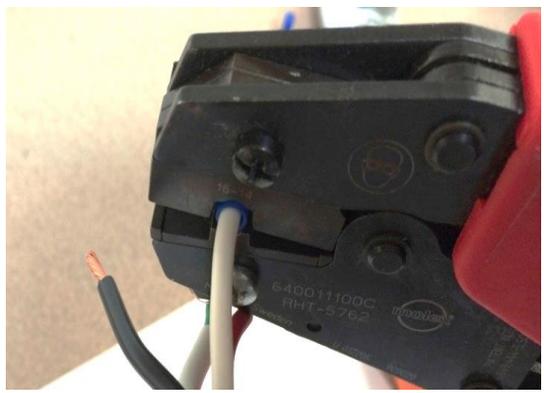

Insert the wire into the quick disconnects; fully seat the wire and crimp.

8

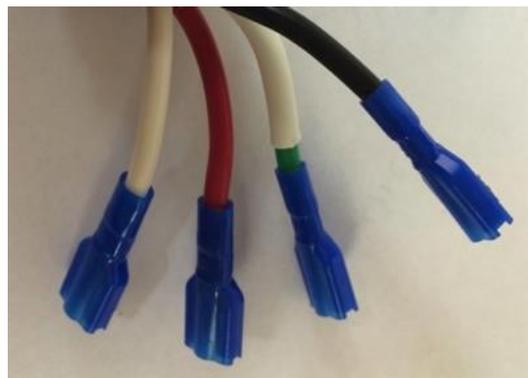

Crimp all QTY (4) wires. Finished assembly shown; quick disconnects crimped to each wire.

9



Connect quick disconnects to the replacement switch.

TERMINAL	WIRE COLOR
1	WHITE
2	GREEN
3	EMPTY
4	BLACK
5	RED
6	EMPTY

10



In the new switch case supplied with the kit, place the switch with markings facing up (as shown) into the cutout of the enclosure. Verify cord is placed on enclosure strain relief (as shown).

11



Place cover of the switch case and insert four corner screws. The replacement remote switch enclosure has the double sided 3M VHB tape preattached.

12



Install remote switch box assembly onto the cart from where it was originally removed. Peel off the paper of the 3M VHB tape and secure remote switch assembly to the cart.

13



Before performing test, verify operation on a known good receptacle. Plug tester into each of the 10 receptacles and compare LED lights to the diagnostic chart printed on the tester. Ensure that the tester indicates a properly wired outlet for each receptacle.