

BioXpress™ Graft Delivery Device - Instructions for Use



Kit components: angled or blunt tip cannula, syringe body and tapered end cap with end cap plug, plunger body with distal vented stylet and stopper, proximal stylet, and funnel.



Prepare DBM product and insert into syringe body.



Apply tapered end cap.



Thread proximal stylet onto distal stylet.



Thread cannula onto tapered end cap.




Depress plunger body to dispense material into cannula.



Depress secondary thumb tab to dispense material to surgical site.



Depress proximal stylet to dispense remaining material to surgical site.

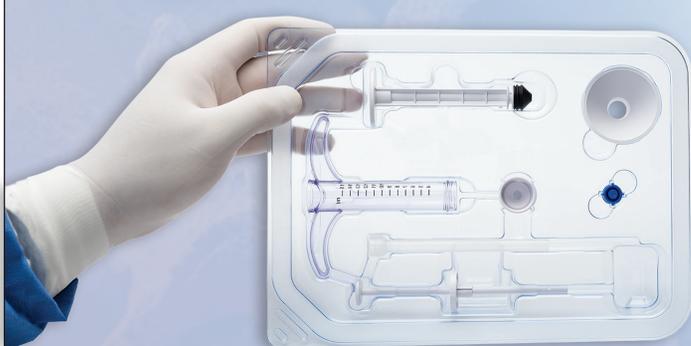

For a full listing of instructions and warnings, always refer to the Instructions for Use packaged with the BioXpress graft delivery device prior to use. (include a link to the document on our website)



Product and Ordering Information

AlloSync™ DBM Gel

1 cc Gel	ABS-2013-01
5 cc Gel	ABS-2013-05
10 cc Gel	ABS-2013-10

AlloSync DBM Putty

1 cc Putty	ABS-2012-01
2.55 cc Putty	ABS-2012-02
5 cc Putty	ABS-2012-05
10 cc Putty	ABS-2012-10

AlloSync CB DBM Paste


1 cc Paste	ABS-2015-01
3 cc Paste	ABS-2015-03
8 cc Paste	ABS-2015-08

AlloSync CB DBM Putty

5 cc Putty	ABS-2014-05
10 cc Putty	ABS-2014-1

BioXpress™ Graft Delivery Device - Vented

Blunt Tip Cannula, 10 cm	ABS-10053-10
Angled Tip Cannula, 10 cm	ABS-10053-10-45
Blunt Tip Cannula, 15 cm	ABS-10053-15
Angled Tip Cannula, 15 cm	ABS-10053-15-45

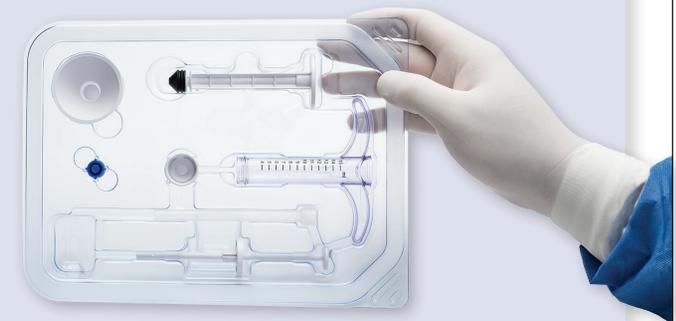

To order, please call: **1-877-255-6727**

Contact your local Arthrex Representative for additional information.

Visit us at www.arthrex.com



Rx only

Instructions for use for the BioXpress graft delivery device contained in this brochure are provided by Arthrex and are intended as reference only. For complete/official instructions for use, please reference the "Instructions for Use" enclosed in the device package. BioXpress is manufactured by Nordson Medical (dba Micromedics, Inc), St. Paul, MN 55121.

AlloSync bone product is indicated for orthopedic applications as filler for gaps or voids that are not intrinsic to the stability of the bony structure. AlloSync bone product is indicated to be packed gently into bony gaps in the skeletal system as a bone graft extender (extremities, spine, and pelvis) and as bony void filler of the extremities and pelvis. These defects may be surgically created or the result of traumatic injury to the bone. Caution: (U.S) Federal law restricts this device for sale by or on the order of a physician.

