



Synergy Vision™ Control Unit Permanent Pairing Setup Instructions

Quick Reference Guide

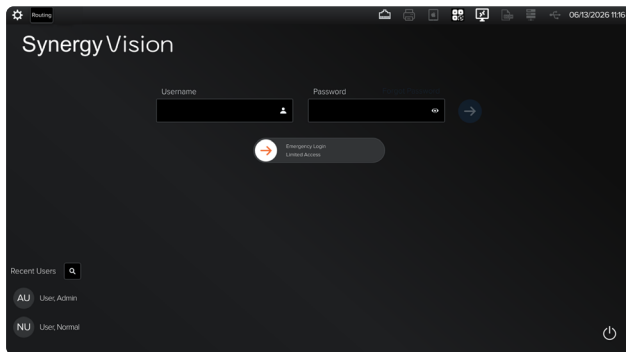


01

Log in to the Synergy Vision console using the admin user account.

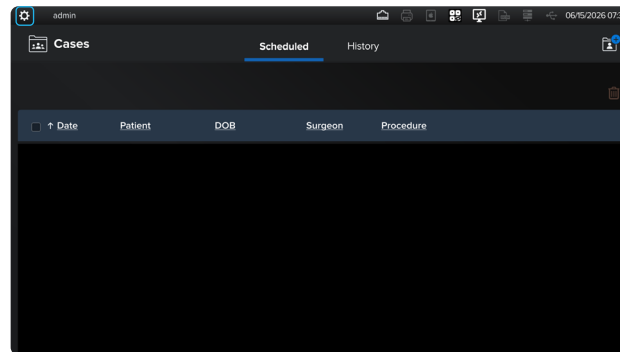


02

Once logged in, tap the gear icon located in the top-left corner to open the All Settings menu.

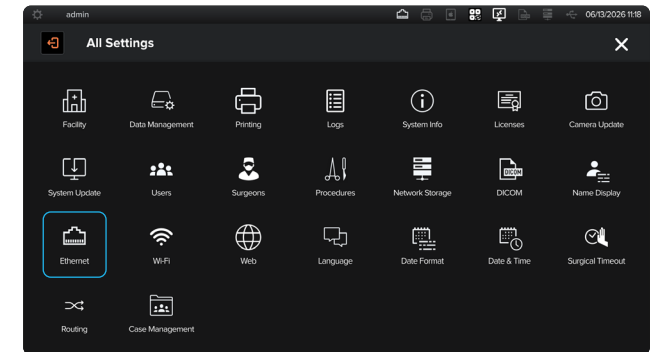


03

On the All Settings page, select the Ethernet icon to view the device's IP address and hostname on the network.

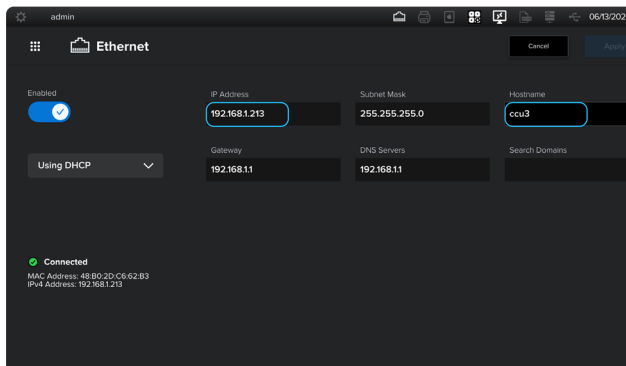


04

This information will be required when pairing the Synergy Vision control unit.

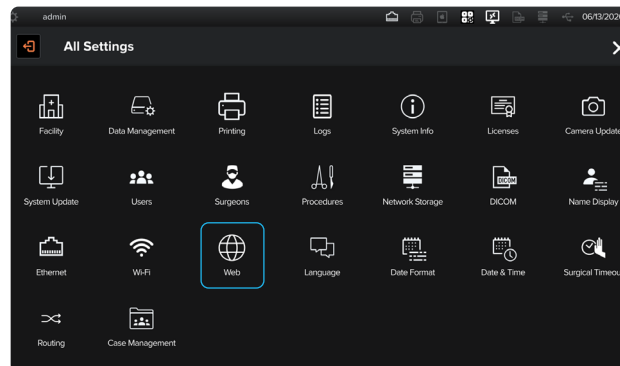


05

Return to the All Settings menu, then select the Web icon.

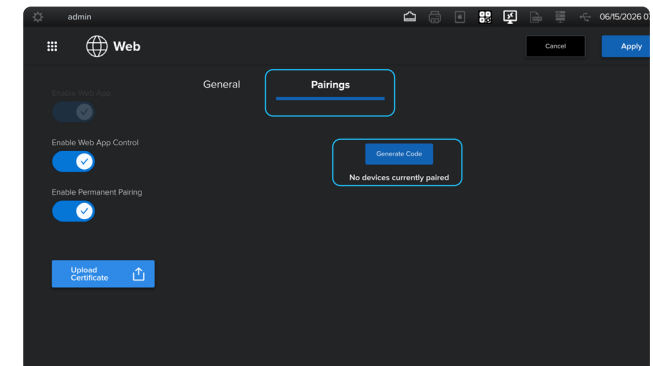


06

Select the Pairings tab, then tap Generate Code.

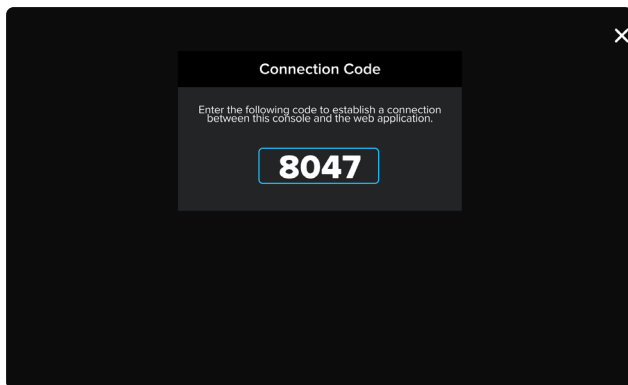


07

You will need the connection code when pairing to the Synergy Vision™ control unit.

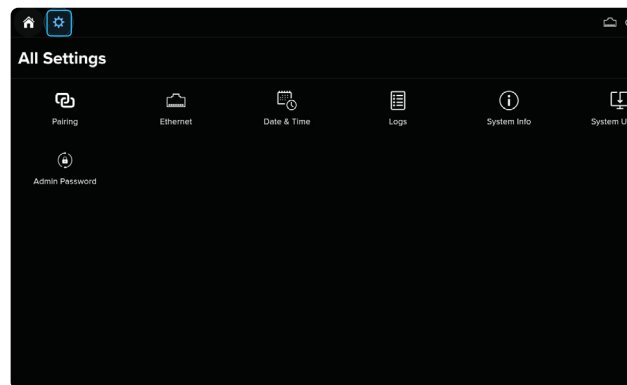

08

Tap the gear icon in the top-left corner to open All Settings menu.

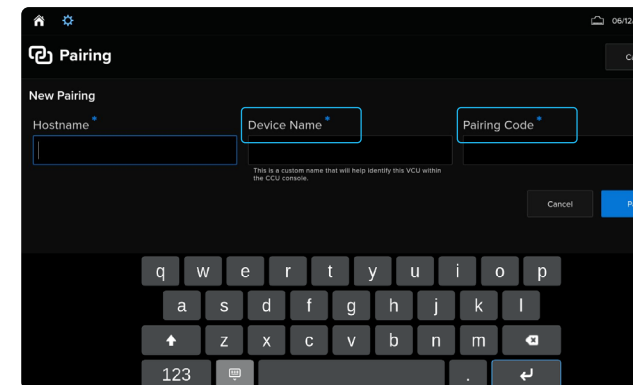

09

On the Synergy Vision control unit, enter the IP address, assign a device name, and then enter the pairing code. Then tap the Pair button.



10

Once successfully paired, a confirmation page will be displayed.

