


RetroButton™ Chart

Welche Looplänge ist die Richtige?

Tiefe des Bohrkanals (mm)	Intraossäre Länge (mm)											Länge des Transplantats im Bohrkanal
	25	30	35	40	45	50	55	60	65	70	75	
20	15	20	25	30	35	40	45	50	55	60	65	15 mm
25		15	20	25	30	35	40	45	50	55	60	20 mm
30			15	20	25	30	35	40	45	50	55	25 mm
35				15	20	25	30	35	40	45	50	30 mm
40					15	20	25	30	35	40	45	35 mm
45						15	20	25	30	35	40	40 mm
50							15	20	25	30	35	45 mm
55								15	20	25	30	50 mm
60									15	20	25	55 mm

RetroButton Looplänge

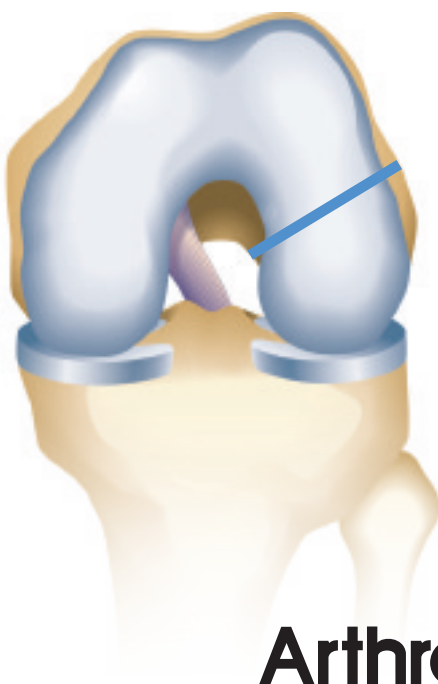

Arthrex®

www.arthrex.de

© Copyright Arthrex Inc., 2007. All rights reserved. LR0186G Vers. A