

Knee Scorpion™


Simplified, Single-Portal Suture Passing and Retrieving



Knee Scorpion
AR-12990



- **Low Profile:** Pass suture in tight spaces with the Knee Scorpion Needle (AR-12990N)
- **Versatility:** Accommodates 2-0 and 0 FiberWire®
- **Efficient:** Pass and retrieve suture in one step through one portal
- **Anatomic:** Angled jaw engineered for the knee joint

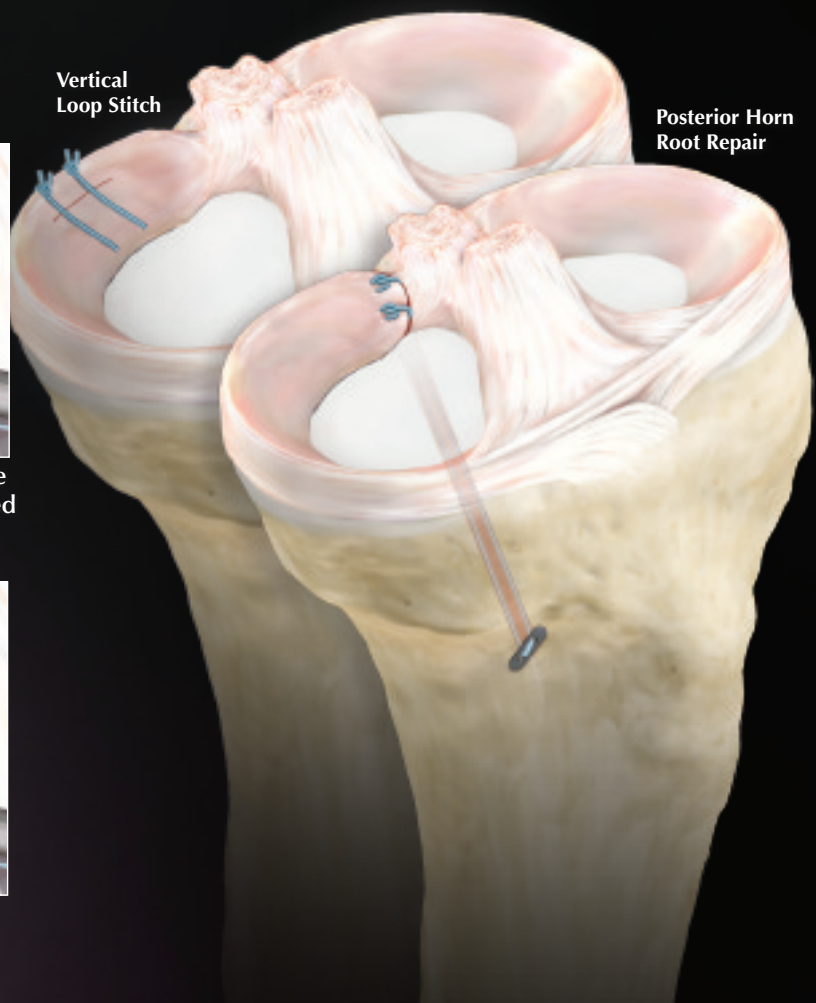

Vertical Loop Stitch

Posterior Horn Root Repair



1

Load 1-2 inches of either 2-0 or 0 FiberWire into the bottom jaw of the instrument and position at desired location.



2

Close the top jaw and squeeze the back trigger to advance suture through the tissue.



3

With the suture captured in the top jaw, retract the Scorpion from the tissue. Release the suture by pulling the suture limbs while squeezing both triggers.



www.arthrex.com

... up-to-date technology
just a click away

View U.S. patent information at www.arthrex.com/corporate/virtual-patent-marking

©2015, Arthrex Inc. All rights reserved.

LS1-00002-EN_A