INSTRUCTIONS FOR USE

DFU-0241-SUB

Rev. 0 06/2020

Disposable Split Adult Return Electrode with Cable (AR-9610SGP)

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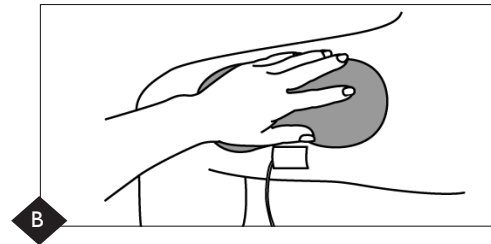
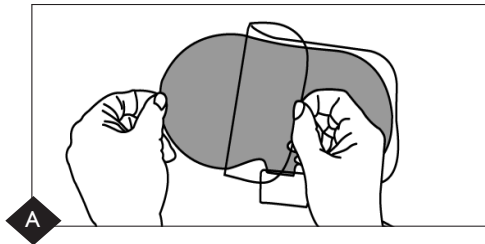
ENGLISH

Instructions for use:

Prepare the electrode application area on patient. Select a smooth, well-vascularized, muscular area as close to the surgical site as possible for pad placement.

- A. Open package and remove electrode.
- B. Completely remove the electrode liner and apply the electrode. Use firm steady pressure to ensure full even contact with the patient.
 - Apply the electrode with the long and non-corded side facing the surgical site.
 - Turn on generator to ensure proper operation. Connect return electrode cord to generator.
 - Remove electrode slowly to avoid skin irritation.

Cautions:

1. If the patient is repositioned after electrode application, ensure the electrode remains in complete contact with the patient and the cord is properly attached.
2. Ensure there are no folds in the electrode.
3. DO NOT prewarm the electrode.
4. DO NOT remove and relocate the electrode after initial application.
5. DO NOT reuse the electrode.
6. To minimize the potential for electrosurgical burns, hair must be removed before placement of the pad.
7. Make sure the entire electrode is flat against the patient's skin, with no potential for tenting under the electrode, as this may lead to ineffective contact with the patient and may cause electrosurgical burns.
8. Do not use the electrode if the pad is discolored or expired.
9. Do not use lubricating jelly on the electrode.
10. Avoid to use warm blanket on the patient.
11. Do not use electrolytic solution.
12. Remove electrode slowly to avoid skin irritation.
13. No latex is used in the manufacturing process.

Federal (USA) law restricts this device to sale by or on the order of a physician.

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