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1 Table of contents

1	Table of contents	12
2	Symbols	12
3	Intended use	13
4	Foreseeable misuse	13
5	Safe application and preparation	13
6	Disassembly	15
7	Assembly	15
8	Cleaning/Disinfection	16
9	Sterilization	19
10	Repair	20
11	Accessories/spare parts.	20
12	Standards/Directives	20
13	Disposal	20
14	Technical data	20

2 Symbols

2.1 Warnings

The following signal words serve to distinguish between warning notes according to the type of danger.

- ➔ **Caution** warns about damage to property.
- ➔ **Warning** warns about physical injury.
- ➔ **Danger** warns about danger to life.

Structure of warning notes:

Signal words

Type and source of danger!

- ⌚ Measure required to prevent danger.

2.2 Other symbols

Notes

Note Note on appropriate use of endoscopes.

Instructions

Structure of instructions:

- ⌚ Instructions on an action to be carried out.

Information on result if necessary.

Lists

Structure of unnumbered lists:

- ➔ List level 1
 - ➔ List level 2

Structure of numbered lists:

1. List level 1
2. List level 1
 - 2.1 List level 2
 - 2.2 List level 2

2.3 Symbols packaging

	Observe operating instructions
	Date of manufacture
	Manufacturer
LOT	Charge designation
SN	Serial number
REF	Order number
WL	Working length
	Non-sterile

3 Intended use

Arthroscopes serve exclusively to visualize the interior of knee joint, shoulder joint, hip joint, ankle joint and wrist during the examination, diagnosis and/or (in combination with endoscopic accessories) treatment.

4 Foreseeable misuse

- ⊖ Do not use arthroscopes for other endoscopic applications.
- ⊖ Connect arthroscopes exclusively to BF devices (body floated devices).

4.1 Contraindications:

- ➔ General inoperability
- ➔ Ambiguous diagnosis
- ➔ Lack of willingness on the part of the patient
- ➔ Technical preconditions not met

5 Safe application and preparation

The endoscope is not sterile packed.

- ⊖ Read and keep the user manual.
- ⊖ Read and keep the user manuals for accessories or medical devices used in combination with the device.
- ⊖ Use the endoscope exclusively as intended, see chapter 3 Intended use.
- ⊖ Make sure that the endoscope is only used by trained and qualified personnel.

- Make sure that the surgeon masters the acknowledged surgical techniques theoretically and practically.
The surgeon is responsible for the appropriate execution of the operation.
- Prior to the first sterilization, clean the endoscope thoroughly (manually or using a machine).
- After every use, clean, disinfect and sterilize the endoscope (see chapter 8 Cleaning/Disinfection).
- Keep the new or unused endoscope in a dry, clean and safe place at room temperature.
- Store and pack endoscopes individually.
- Store endoscopes in a dry place at room temperature.
- After every cleaning/disinfection and prior to every use, check the endoscope for: cleanness, function and damages, e. g. insulation, loose, bent, broken, chapped, rough, worn, broken off parts etc..
- Do not use a damaged or defective endoscope. Replace damaged parts by original spare parts immediately.
- Reject damaged endoscopes immediately.
- To prevent damages on the working end: Insert the endoscope carefully through a trocar.
- Do not bend the endoscope.

It is possible that light with a high radiation energy emerges from the viewing window and raises the temperature of the tissue.

- Make sure that the surface temperature of the body does not exceed 41 °C.
- During longer applications, let the body cool down if necessary.
- Use light sources with max. 300 W (xenon) or 250 W (halogen).
- Check image quality (clear and free of distortion) and light transmission through the glass fibers.
→ Hold the light guide connector against non-glare light.
Should the glass fibers appear as black spots on the distal end (broken glass fibers), it is possible that the light output is insufficient.
- To maintain the optical quality, make sure that the endoscopes are not exposed to gamma rays.

Note In case of suspected or diagnosed Creutzfeldt-Jacob syndrome, ARTHREX recommends not to reuse the endoscope but to dispose of it.

- In case of suspected or diagnosed Creutzfeldt-Jacob syndrome, immediately take measures to prevent the infection of other patients, users or third parties.

- Observe country-specific laws and specifications.

6 Disassembly

Disassemble the endoscope as follows:

- Disassemble the light guide connector (see Fig. 1).

7 Assembly

7.1 Assembly

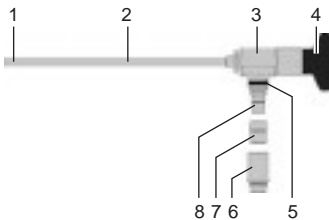
Fig. 1 Construction

- (1) Objective with sapphire window
- (2) Duct
- (3) Body
- (4) Eyepiece
- (5) Colour code ring (line of vision)
- (6) Adapter for light guides, removable (Storz or Olympus)
- (7) Adapter for light conductors, removable (Wolf)
- (8) Connecting piece for light guides (ACMI)

Note Adapter for light guide Storz is included by default. Adapter for light guides Olympus is optionally available.

Colour code for indication of line of vision:

green	0°
blue	12°, 15°
red	30°
orange	45°
yellow	70°
black	90°

For a safe assembly:

- Always hold the endoscope on the body or the eyepiece.
- Make sure that the adapters of the light-conducting cables comply with the adapters of the endoscope (see drawing).
- Make sure that glass surfaces do not interact with other instruments.

7.2 Combinations

Danger

Danger to life due to high tensions and currents!

- ⊖ Make sure that the leakage currents of the patient are minimized for combinations.
- ⊖ Connect arthroscopes exclusively to BF devices (body floated devices).
- ➔ Combinations with medical devices with energy-operated, endoscopically applicable accessories:
- ➔ To prevent electric coupling between the patient and the device, ARTHREX recommends the use of ARTHREX devices and accessories.

Combine endoscopes with other medical devices if:

- ➔ the intended use in the instruction manual or the operating instructions allows it.
- ➔ the technical data in the instruction manual or the operating instructions allow it.
- ➔ the standard of the TV objectives or cameras complies with the general standard.

8 Cleaning/Disinfection

8.1 After every use

Caution

Damage to property due to incorrect cleaning!

- ⊖ Do not clean endoscopes in an ultrasonic bath.
- ⊖ Do not gamma-sterilize endoscopes.
- ⊖ Sterilize contaminated endoscopes as soon as possible.
- ⊖ Put encrusted endoscopes in 3% H₂O₂ for 5 minutes prior to cleaning and disinfection.
- ⊖ Use completely desalinated water.
- ⊖ In case of machine cleaning, put endoscopes on suitable screen baskets fit for cleaning (make sure there are noncleaned areas).
- ⊖ Make sure that the fixating elements hold the endoscopes securely.
- ⊖ Take apart dismountable endoscopes (see chapter 6 Disassembly).

- Connect the flow or working canals of the endoscopes with the flushing connectors of the washer.
- Preferably dispose of dry.
- In case of wet disposal, use cleaning-active disinfection agents. Prior to machine cleaning and disinfection, clean endoscopes thoroughly with clear, flowing water.
- Clean manually or using a machine. Mind manufacturer's indications (e.g. dosage).

8.2 Manual cleaning/disinfection

- Clean the endoscopes manually exclusively if machine cleaning is not possible.
- Mind the indications of the cleaning agent manufacturer.
- Use cleaning agents with medium pH value (5 - 11).
- If immediate cleaning is not possible:
Put endoscopes in enzymatic cleaning agent for 15 minutes, so that all surfaces, cavities, lumens, and openings are covered. Mind the instructions of the disinfection agent manufacturer.
- Do not exceed the maximum exposure time of 15 minutes.
- Clean the windows (distal and proximal) and glass fiber ends with a cotton swab (wood) or a soft cloth soaked in isopropyl alcohol (70%) or acetone. Alternatively, use neutral detergent (hand soap).
- Always rinse sufficiently and thoroughly with clear flowing water after chemical disinfection. Mind the instructions of the disinfection agent manufacturer.
- Remove adhesive material with a soft plastic brush. Do not use abrasive cleaning agents or metal brushes.
- Clean lumens or canals with soft round plastic brushes.

Note The diameters of the lumen and the brush have to accord.

- Use completely desalinated water for the final rinsing.
- Dry the endoscopes with an absorbant, soft and lint-free cloth.
- Dry endoscoped at 60 °C for at least 2 hours.
- Dry lumens and canals with compressed air.

8.3 Machine cleaning/ disinfection

- ARTHREX recommends machine cleaning of endoscopes.
- Clean endoscopes exclusively in industrial washers with suitable cleaning method.

- Mind the indications of the washer manufacturer and the cleaning agent manufacturer.
- Make sure that the washer manufacturer and the cleaning agent manufacturer recommend their methods for endoscopes.
- Use cleaning agents with medium pH value (5 - 11).
- Put endoscopes in enzymatic cleaning agent for 15 minutes at most.
- Apply thermally neutral methods in the pH-neutral range (e.g. enzymatic cleaning agents).
- For the selection of the washing cycle, mind the material (e.g. stainless instrument steel, aluminium) of the endoscopes that are to be cleaned. Mind the instructions of the device manufacturer.
- Pre-rinse endoscopes twice.
- Clean endoscopes at 40 °C - 60 °C.
- Disinfect at 80 °C - 95 °C.
- Dry at 50 °C.
- Observe sufficient drying time.
- Make sure that there is no residue on the glass fiber surfaces.
Burning in with a light source can impair the light transmission.
- Remove the endoscopes from the machine immediately after the cycle is completed.
- Let the endoscopes cool down to room temperature (23 ± 2 °C).

8.4 Care, checking

- Let the endoscopes cool down to room temperature.
- Slightly grease movable parts (e.g. joints and ends) with sterilizable and vapour-permeable protective oil.
- Assemble/dismountable endoscopes (see chapter 7 Assembly)
- After every cleaning and disinfection, check endoscopes for: cleanness, function and damages, e. g. insulation, loose, bent, broken, chapped, rough, worn, broken off parts etc.
- Reject and replace damaged and defective endoscopes immediately.
- Endoscopes are maintenance-free.

8.5 Packing

- Store and pack endoscopes exclusively individually.
- Store endoscopes in original packaging or in a sterilization container.
- Store endoscopes with a small diameter (3.0 mm and smaller) in the protective sleeves for transport and storage available at ARTHREX.

9 Sterilization

ARTHREX endoscopes are compatible with all established sterilization methods.

The operator is responsible for the sterility of the endoscopes.

- Apply exclusively suitable methods for sterilizing endoscopes.
- Make sure that a sufficient number of sterile endoscopes are available.

When sterilizing, please note the following:

- Prior to sterilizing, clean the endoscopes thoroughly (see chapter 8 Cleaning/Disinfection).
- Make sure that sterilization medium reaches the whole surface.
- Avoid contact with metal surfaces.
- Make sure that the fixating elements hold the endoscopes securely.
- Do not sterilize endoscopes with protective transport sleeve.
- Sterilize endoscopes individually packed in suitable packagings.

9.1 Sterilization process

- Sterilize endoscopes according to the generally accepted hospital method.
- Mind the indications of the manufacturers of the used auxiliaries.

Validated sterilization method:

Parameters	134 °C/273 °F Double fore-vacuum
Sterilization time	5 - 20 min.
Packaging	In metal sterilization containers, paper laminate sterilization bags, type Steriking

*Note Vapour-sterilize exclusively endoscopes marked with **Autoclav/autoclavable**.*

Damages due to incorrect cooling!

- Do not accelerate the cooling of endoscopes. This causes damages and reduces the durability.

After the sterilization:

- Let the endoscopes cool down to room temperature.
- Open the packagings after the cooling.

10 Repair

To guarantee the operational safety of the endoscope optics:

- Let exclusively the respective dealer or a repair facility authorized by ARTHREX carry out repair works.
- Use exclusively original spare parts for repair works.

Note The guarantee and warranty claims expire if repair works are not carried out by repair facilities authorized by ARTHREX.

Prior to return:

- Clean and sterilize endoscopes.

11 Accessories/spare parts

Our catalogs contain information about accessories and spare parts.

12 Standards/Directives

Sopro-Comeg manufactures all endoscopes in accordance with the following directives:

- ➔ Medical device directive 93/42 EWG last changed by 2007/47/EG
- ➔ DIN EN ISO 13485
- ➔ Series of standards IEC 60601
- ➔ Series of standards ISO 8600
- ➔ FDA regulations (FDA Quality System Regulation 21 CFR Part 820)
- ➔ other product- and material-specific standards and regulations

13 Disposal

For the disposal, mind the following:

- Dispose of the packaging and used parts in accordance with the country-specific regulations.
- Protect the device from access by unauthorized persons.

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© 2011
Arthrex, Inc.
Arthrex Med. Inst. GmbH
Sopro-Comeg GmbH